MICRO.MEDICA
FIBER POSTS

The logo features a stylized 'm' composed of multiple parallel lines, with a solid green circle in the center. Below the graphic, the text 'MICRO.MEDICA' is written in a bold, black, sans-serif font, and 'FIBER POSTS' is written in a bold, green, sans-serif font.

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Technology, Innovation and a winning team are the necessary elements to achieve cutting-edge and high-quality products. In Palestro, in the heart of Lomellina Pavese, there is a company capable of transforming ideas and desires into reality. The dedication to work and the seriousness are the ingredients that have seasoned almost thirty years of passion spent working in the dental sector, carefully designing and manufacturing high quality products exported all over the world.

THIS IS OUR MISSION.

We are specialized in the creation of custom products, we transform your ideas in special objects even in very small series, we provide our experience to create new products or product lines, with care and competence of the artisans of the past.

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We work with respect for mankind, animals and the environment. We choose for all our products recyclable packaging, we produce in our factory with great attention in the choice of suppliers and raw materials in compliance of the green philosophy of our firms.

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Brushing Water Treatment is the new exclusive surface treatment system developed by Micro Medica to increase the retentive power fiber posts. The Brushing Water treatment is an innovative system that makes the surface of the posts extremely rough in fiber without damaging them. A very light massage prepares the surface of the posts in a homogeneous way, increases their retentive power until 40% more than an untreated post. The system also guarantees thorough the cleaning of the posts, eliminating any impurities from the fiber. The Brushing Water system guarantees perfect adhesion with any cement, favoring an incredibly resistant cement-fiber bond. Look for the BWT brand on the packages of the posts which guarantees the treatment of the posts you have purchased.

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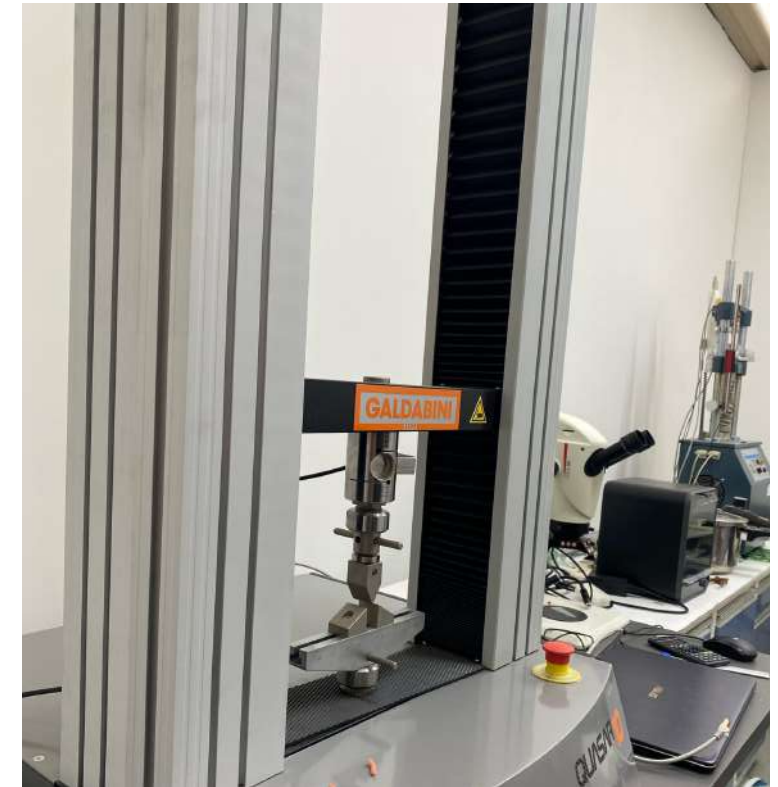
Selected Raw Materials and Rigorous Quality Control All of our products are made using high-quality raw materials, carefully selected from the market and certified for medical use. Upon arrival at our facility, each batch of raw material undergoes a series of thorough inspections and tests, as part of our quality management system. This process ensures that every stage of production begins with a reliable, safe, and compliant foundation—meeting the highest standards in the industry.

Internal Testing Laboratory

Our company has an in-house laboratory dedicated to materials testing, guaranteeing high standards of quality and reliability. The laboratory is equipped with sophisticated machines for precise mechanical analysis and the evaluation of all surface treatments. This advanced equipment allows us to carefully monitor material performance and ensure consistent, high-quality results at every stage of production.

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High-Precision Production and Certified Quality;

Our posts are manufactured using state-of-the-art CNC machines, ensuring maximum precision throughout the entire production process. To maintain consistently high-quality standards, we perform microscopic and dimensional checks approximately every 2 hours directly on the production line.

Certified Surface Treatment;

After production, each post undergoes an innovative cleaning process known as the Brushing Water Treatment, a gentle treatment with soft bristles designed to enhance the surface roughness of the post. This procedure follows a validated operational protocol, certified by an external laboratory, which has confirmed its effectiveness in bacterial decontamination.

Random Sample Testing;

Randomly selected batches from different production runs are subjected to in-depth testing to ensure full compliance with mechanical and functional requirements.

Innovation, Evolution & Customization;

Our posts are available in roughened and silanized versions, ensuring excellent adhesion and clinical integration.

- **Continuously evolving products, designed to adapt to the needs of dental professionals.**
- **Exclusive productions, developed to meet the specific demands of specialized markets.**
- **Custom-designed burs for each type of post, ensuring perfect compatibility between tool and component.**

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Some general principles and guidelines:

1. The post should not extend too far apically (to avoid the risk of perforation) nor too little (otherwise it will not provide support).
2. A 2-4 mm apical flap (i.e., part of the root gutta-percha) should remain to maintain a good apical seal.
3. The minimum thickness of the residual dentin around the post must be sufficient (ideally ≥ 1 mm) to avoid root fractures.
4. The post must be centered in the canal, avoiding inclinations that risk stressing a thin root wall.
5. If the root is curved or has anatomical restrictions, the length of the post can be adjusted, always keeping in mind mechanical strength and preservation of the tooth structure.
6. The post must be "prosthodontically guided": the ideal position of the prosthetic restoration (crown) should guide the choice of the post axis.

General guidelines:

The choice of post type depends on the shape and diameter of the canal, root length, and tooth type.

- Incisors and Canines: Preferably Simplex, Cobra (CBC), or Hi.Tecno XF.
- Premolars: HS, Rapid Post, or Medix Post for a balance between taper and strength.
- Molars: Cobra (CB) or 911 Series in main canals, even in multiple combinations. Always maintain ≥ 4 mm of apical gutta-percha and ≥ 1 mm of residual dentin.

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From the crystalline purity of the quartz comes our new incredible Transparent fiber post
The Light finally meets no Obstacles !!

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Everyday with great passion we dedicate ourselves to the creation of increasingly advanced products and technology to meet the needs of a ever-changing market. Technology and a Winning team are the ingredients to manufacture incredible products and to help our customers win the challenge with the market of the future. Crystal quartz fiber is the result of years of research, a new production method allows to obtain a raw material of the highest level with exceptional mechanical characteristics and an almost crystalline transparency.

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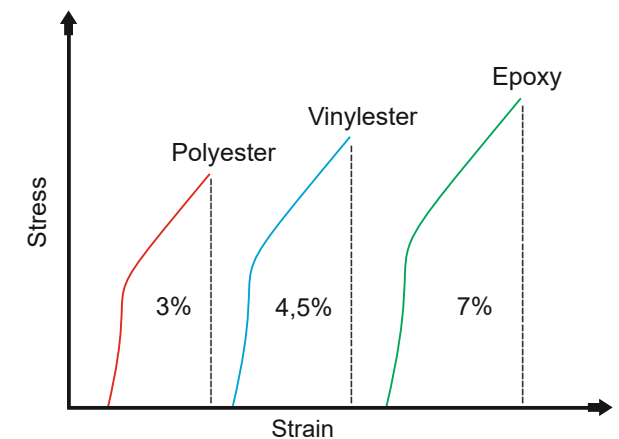
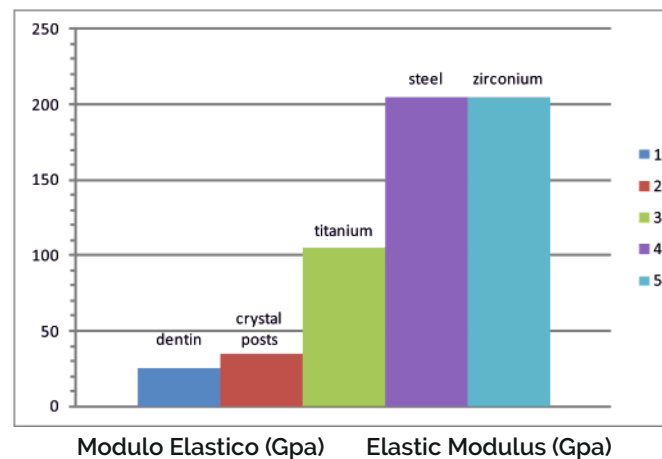
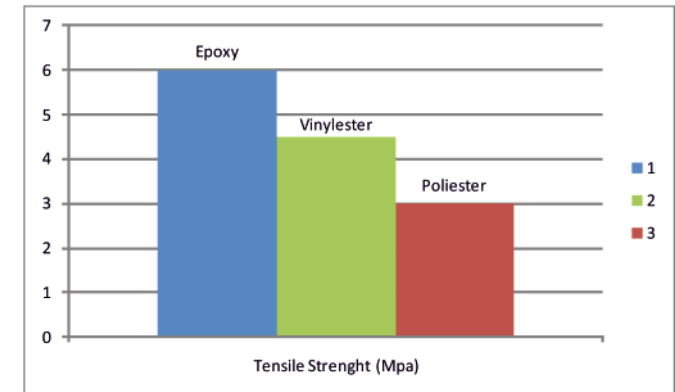
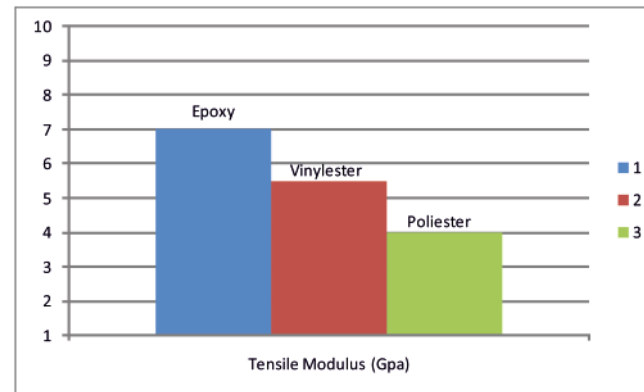
Micro Medica's new transparent quartz fiber post, Crystal, is the result of many years of research and experimentation, the use of pure raw materials and a cutting-edge production technology have made possible to obtain a product with unique characteristics. Using a perfect mix of quartz fiber and epoxy resin we have created a post with incredible transparency and exceptional mechanical characteristics.

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The pre-tensioned longitudinal fibers and an elastic modulus very similar to the one of dentin make Crystal posts the instrument suitable for resistant and performing reconstructions.

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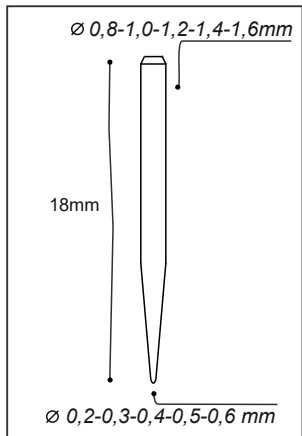
The light finally meets no obstacles and can pass through the pin without problems, this extremely important quality guarantees to the professional perfect cementations. Every single shape of the new pins has been designed to meet the needs of modern endodontics while respecting the anatomy of the root canal. Crystal posts, like all posts made by Micro Medica, are superficially treated with the new exclusive BWT Brushing Water Treatment.

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
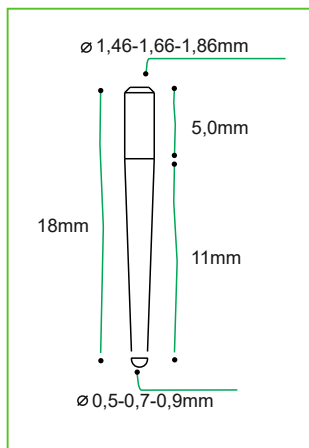
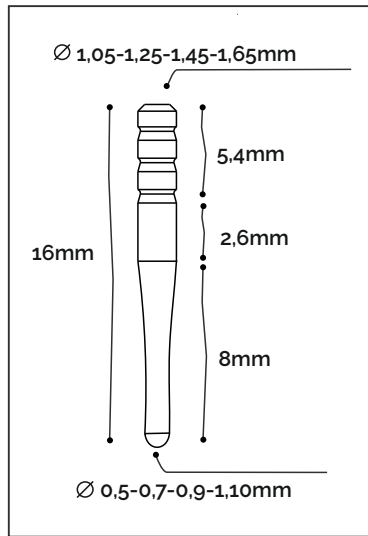
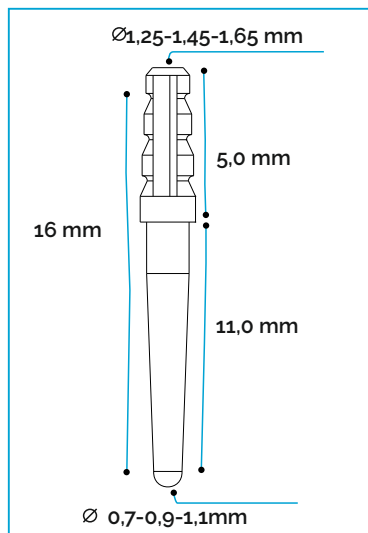
The new Red Point fiber posts are the evolution of traditional fiber posts, the use of high quality raw materials and cutting-edge technology have allowed the creation of this new line of posts. Transparency and a decidedly high mechanical resistance distinguish this new product. The Red Point posts contain a colored core that becomes a really useful element if it's necessary to reopen the root canal: by following the red line we will be able to go all the way to the root canal without any risk. The colored core of the new Red Point posts is made using the same fiber of which the posts are made, in this way the core is not an element of a different nature and the mechanical characteristics of the post remain unaltered.

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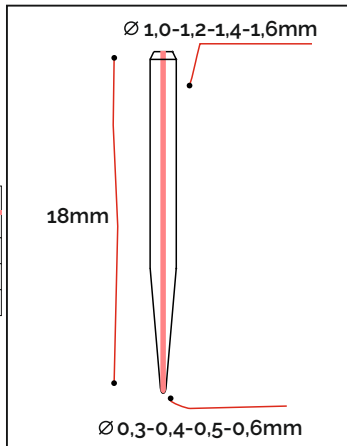
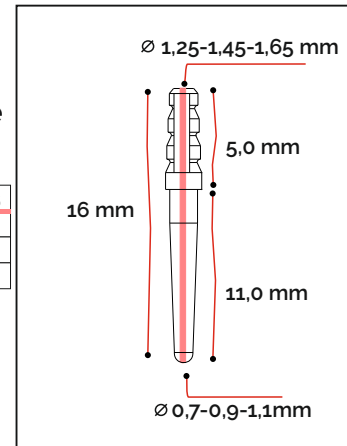
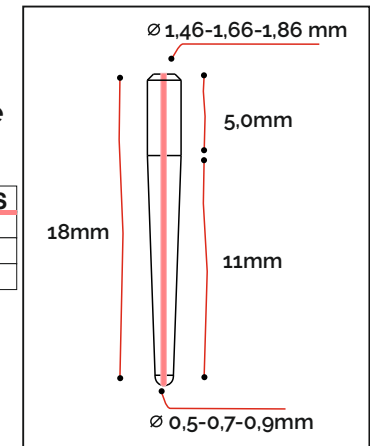
Thanks to the new Nano materials, Micro Medica has created a new generation of super technological fiber posts.

The new Nano Ceramic fiber posts take advantage of the exceptional characteristics of this new technology: the posts made with this new material have mechanical resistance characteristics superior to traditional posts. By making the most of the possibilities of Nano particles we have succeeded to obtain posts with high radio opacity. The light passes through the post and allows a good photo-curing of the cements directly into the root canal, in this way highly performing cementations are obtained.

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

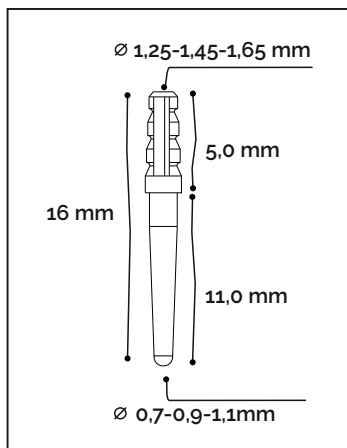
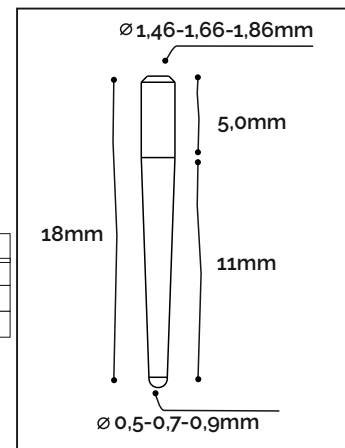
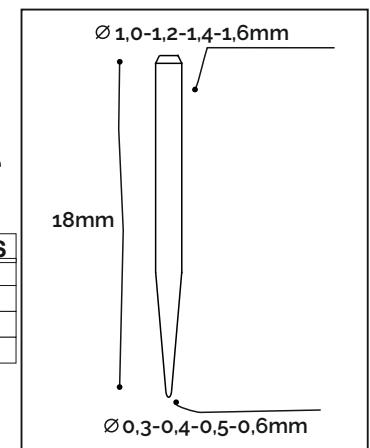
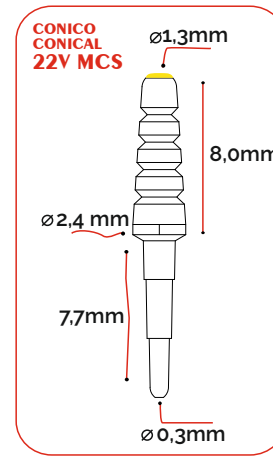
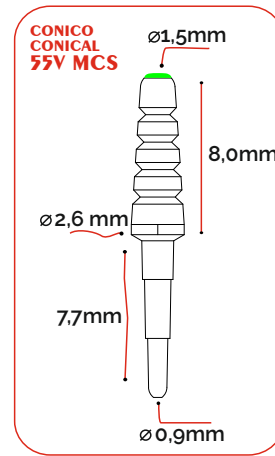
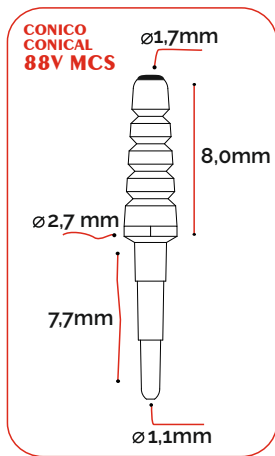
Microcore ESSE is the new system of conical and cylindrical glass fiber endocanalary posts with integrated abutment, a simple and ergonomic system for creating perfect reconstructions.

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The ease of use of the ESSE

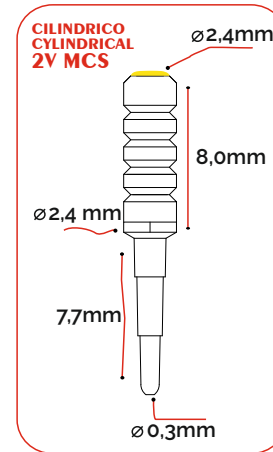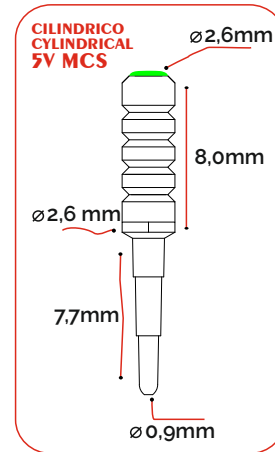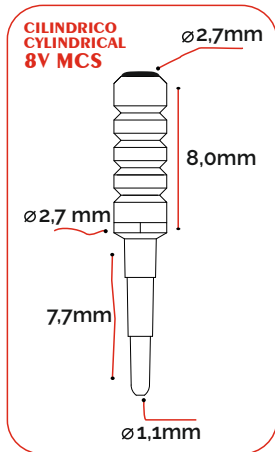Microcore posts allows them to be used in all those cases where it is necessary to build resistant abutments with technical characteristics that can guarantee a valid support to the prosthetic works they will have to support.

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The particular shape of the new pins allows a homogeneous distribution of chewing loads, the size of the apical area guarantees this an optimal fit. Microcore ESSE pins can be easily milled using the normal diamond burs FG or CA until obtaining the desired abutment shape. The ESSE Microcore system and indispensable for the realization of homogeneous and resistant structures.

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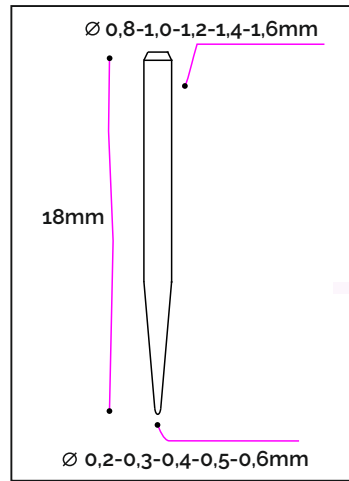
Simplex is the modern evolution of the classic cylindrical fiber posts, a new design features these posts. The elongated shape is extremely contained especially in the apical area allows you to place this post easily even in narrow and tapered roots.

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For those who want to have all five posts sizes Simplex is available in a new kit that contains 5 posts for each size and the related drills. Each post head is colored with the same color as the drills to facilitate the matching.

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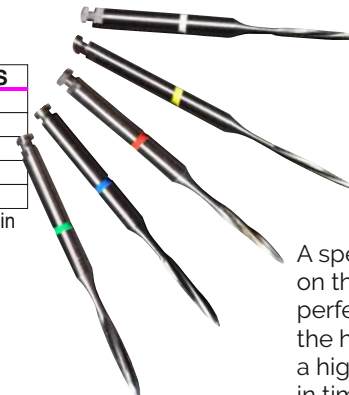
A special drill is available for each diameter, calibrated on the exact size of the post, the drill post coupling is perfect. Micro Medica drills are produced using the highest quality stainless steel, that guarantees a high cutting power and excellent duration in time.

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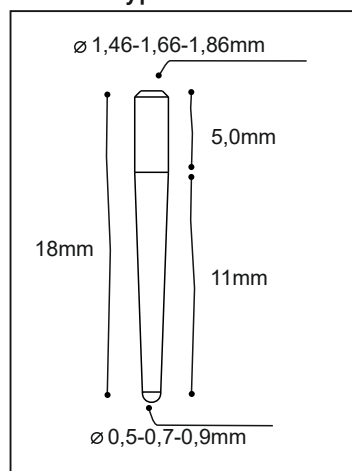
The new conical post of the HS line was designed following the requests of many dentists who were looking for a conical post with small dimensions and with a extremely small diameter in the apical part, thanks to its taper the HS post adapts easily to any root canal. HS is the ideal tapered post that can be used in most clinical cases without problems.

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For those who want to have all three posts sizes HS is available in a new kit that contains 5 posts for each size and the related drills. Each post head is colored with the same color as the drills to facilitate the matching.

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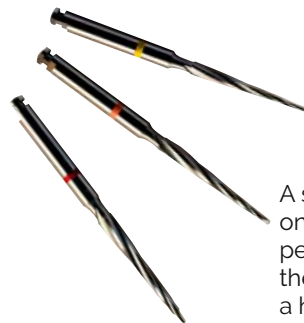
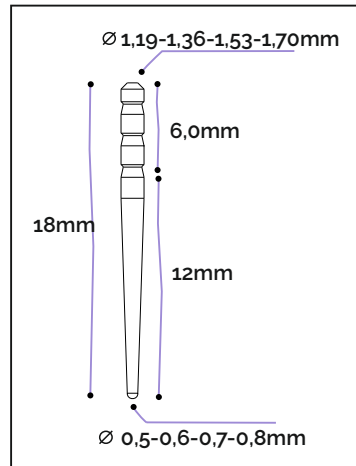
The evolution of root canal preparation techniques and filling materials has created the need for minimally invasive posts that in terms of shape and anatomy could somehow resemble the ones of natural root canals. The conical posts of the MediX Post line meet these characteristics and are the evolution of the traditional conical posts.

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For those who want to have all four posts sizes MediX Post is available in a new kit that contains 5 posts for each size and the related drills. Each post head is colored with the same color as the drills to facilitate the matching.

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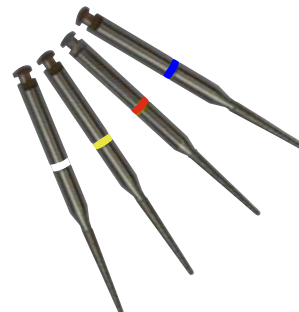
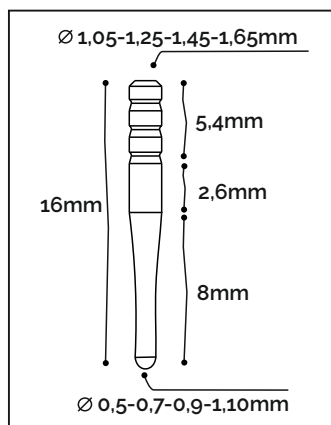
The anatomical shape of the new 911 Series post has been designed to facilitate the insertion even in channels that have particular shapes and sizes. The anatomical apical part adapts perfectly to any root canal.

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For those who want to have all four posts sizes 911 Series is available in a new kit that contains 5 posts for each size and the related drills. Each post head is colored with the same color as the drills to facilitate the matching.

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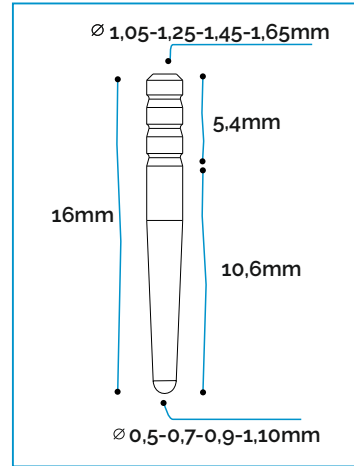
With an extremely technical design, the posts of the Rapid line collect a series of innovative features that make them truly unique. A wide series of retentions placed in the extra coronal part and the conical shape, allows the reconstruction materials and cements to create a perfect and long-lasting bond.

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For those who want to have all four posts sizes Rapid Post is available in a new kit that contains 5 posts for each size and the related drills. Each post head is colored with the same color as the drills to facilitate the matching.

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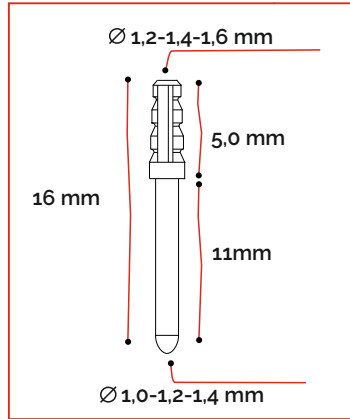
For the construction of abutments with large diameters is often necessary to use large quantities of composite material that is sometime supported by posts of inadequate diameter. To avoid the failure of this type of reconstruction it is advisable to support them with posts with a proportionate diameter and above all with the coronal part of the right size. Ideal for this type of reconstruction are all the posts of the Cobra line.

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For those who want to have all three posts sizes Cobra is available in a new kit that contains 5 posts for each size and the related drills. Each post head is colored with the same color as the drills to facilitate the matching.

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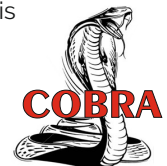
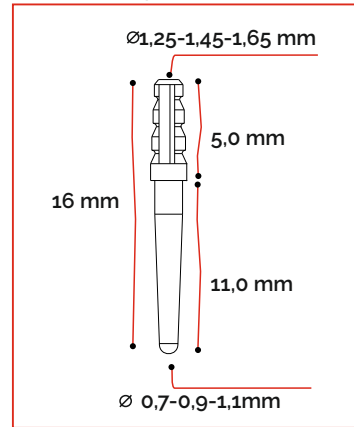
For those who want to have all three sizes of posts Cobra is available in a new kit that contains 5 posts for each size and the related drills. Each post head is colored with the same color as the drills to facilitate the matching.

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

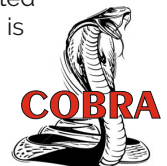
A special drill is available for each diameter, calibrated on the exact size of the post, the drill post coupling is perfect. Micro Medica drills are produced using the highest quality stainless steel that guarantees a high cutting power and excellent duration in time.

hi.techno

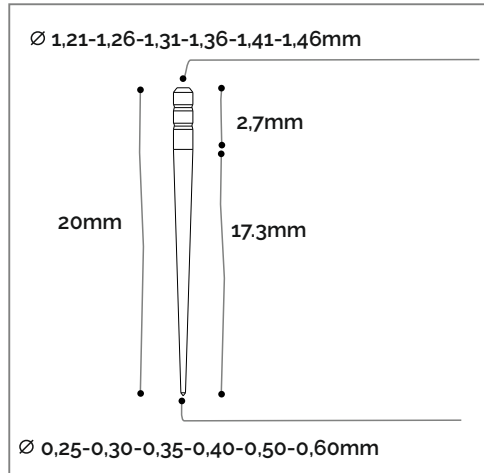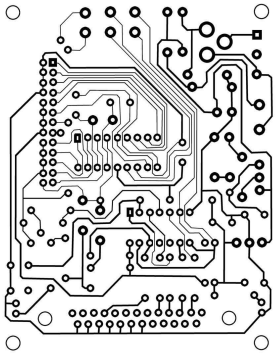Conico Extra Fine Extra Thin-Conical

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Hi Techno is the brand new Micro Medica's post made to be used in combination with cements for root canal closure or gutta-percha, its form is compatible with the majority of endocanal instruments on the market. Extremely thin, the tip of these posts comes in six different sizes, it has good flexibility especially in smaller sizes and is a valuable aid in apical sealing with both gutta-percha and traditional cements.

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For those who want to have all six sizes of posts Hi Techno is available a new kit that contains 5 posts for each size. Each post head is colored with the same color as the drills to facilitate the matching.

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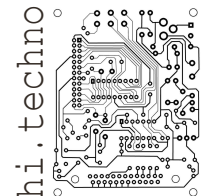
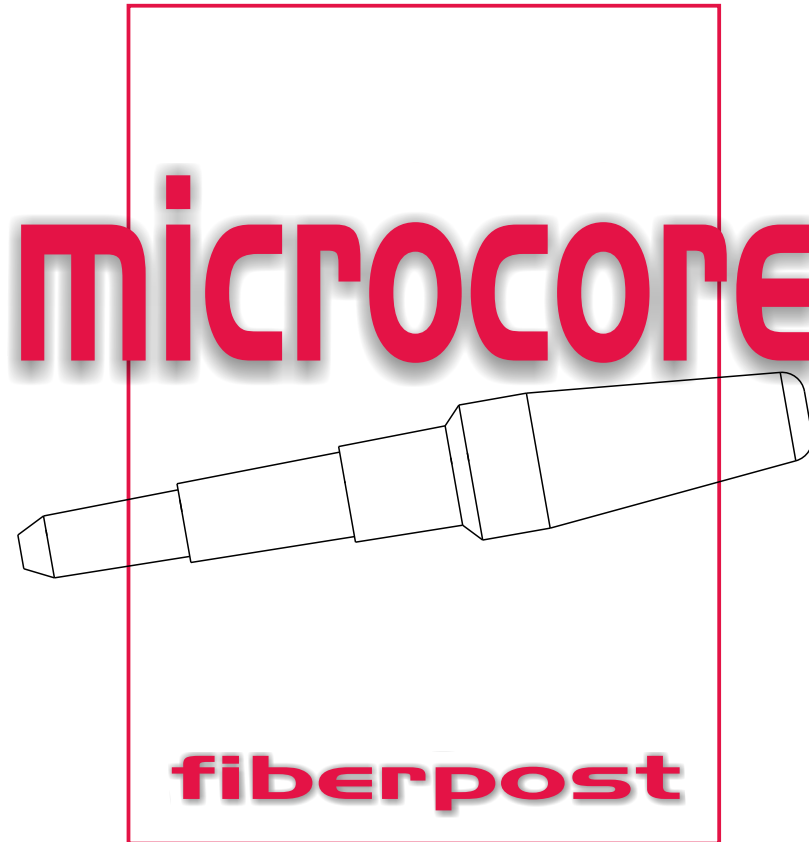
Microcore is the new system of conic endocanal post in fiberglass with Integrated Abutment made by Micro Medica, a simple and ergonomic system for the realization of perfect reconstruction. The practicality of use of Microcore post allows its use in all those cases where it is necessary to build resistant abutment with technical characteristics that can guarantee a valid support of the prosthetic work. Microcore post are produced with a special mix of translucent fiberglass and highly resistant epoxy resin, capable of withstanding important loads of mastication. The translucency of the new fiber is a valuable aid in the process of cementing. Nine conic posts of different sizes form the Microcore system, nine sizes calibrated respecting the size and morphology of root canals. Each post is marked with a color that ease the use and identification. The particular shape of this new posts allows a homogeneous distribution of the mastication force and the small size in the apical zone ensures an excellent adaptation. The drills for the preparation of the root canal has an innovative design, thanks to this, the water cool the drill and the root canal, in this way a perfect milling without trauma is obtained. Each drills is marked by a colored ring of the same color with which the posts are marked in this way find the right drill for each post is extremely easy and intuitive. The Micro Core posts can be milled using standard diamond burs, CA or FG, to obtain the desired shape of the abutment. The Micro Core system is indispensable for the realization of homogeneous and resistant structures.

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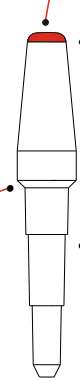
Ideal conicity designed to offer secure and consistent support to any type of reconstruction.

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Base of the abutment inclined for a better adaptation inside the root canal.

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The diameter in the apical zone is very small from 0,7mm to 1,1mm max, the post adapts easily to any root canal


Conical abutment integrated and proportionate at the length of the post - available in 3 different diameters - easy to mill thanks to the BWT treatment the surface has a great retentive power the composite adhere perfectly - excellent mechanics resistance

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Cylindrical emergency diameter from 2.0 mm to 2.7 mm to ensure maximum strength

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



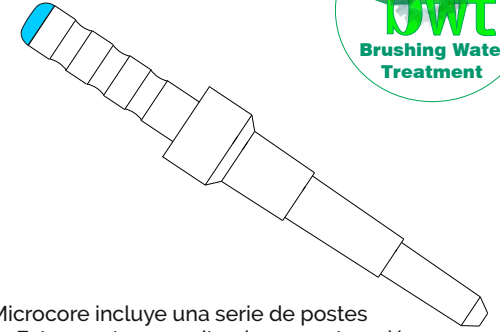
Endodontic part made with conical cylinders, one above the other, this structure allows to discharge evenly the mastication forces

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









The microcore line includes a series of posts without abutment, these posts allow tooth reconstructions of partially decayed teeth. These posts can also be used as normal posts in the daily routine of the dental studio, while maintaining all the technical characteristics of the posts with the abutment.

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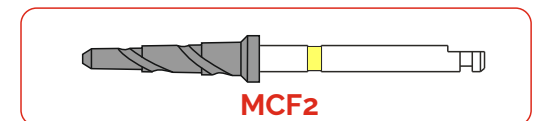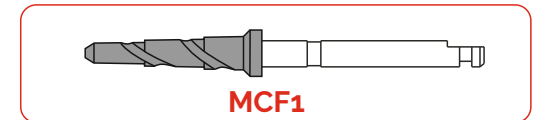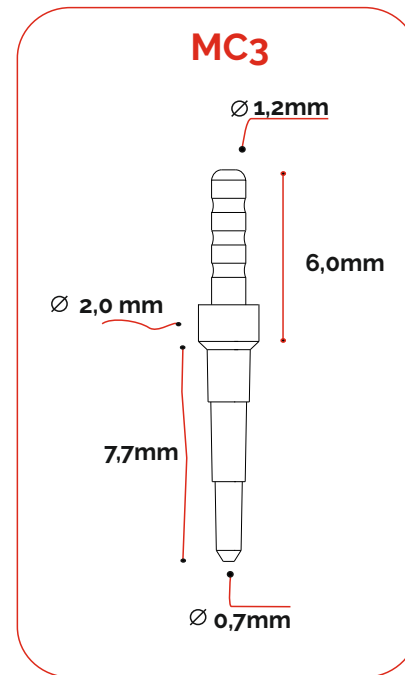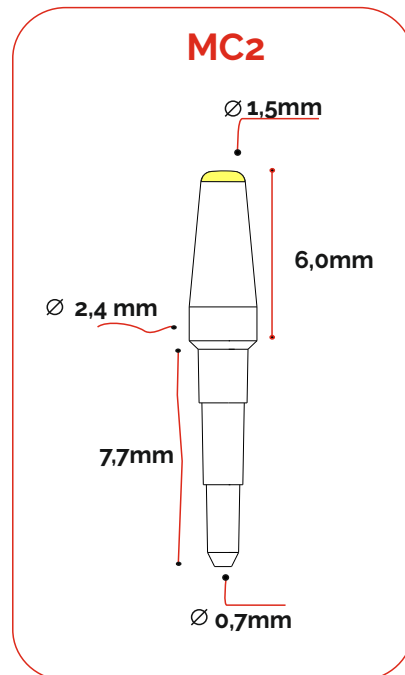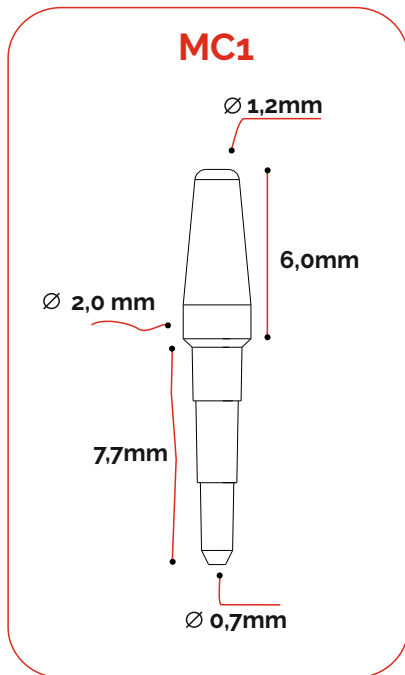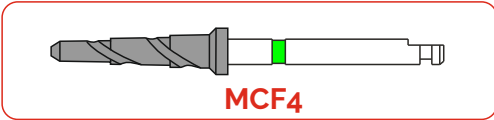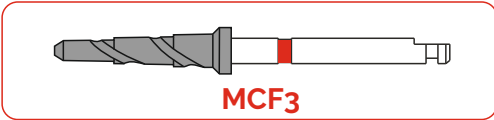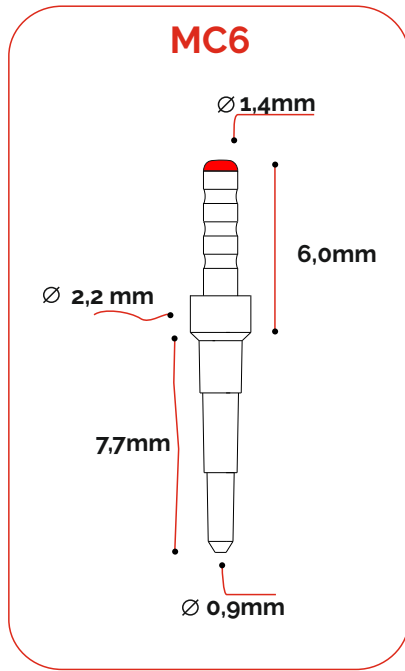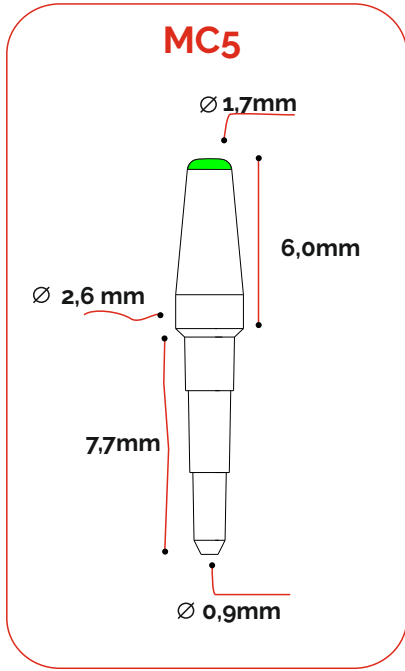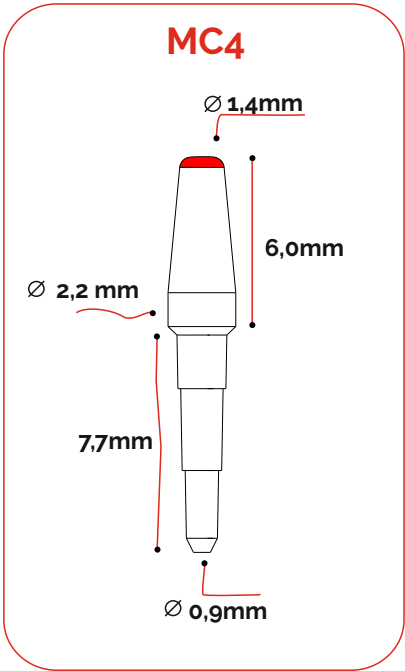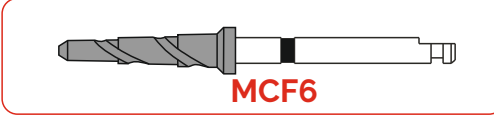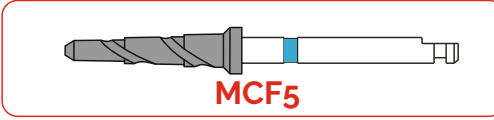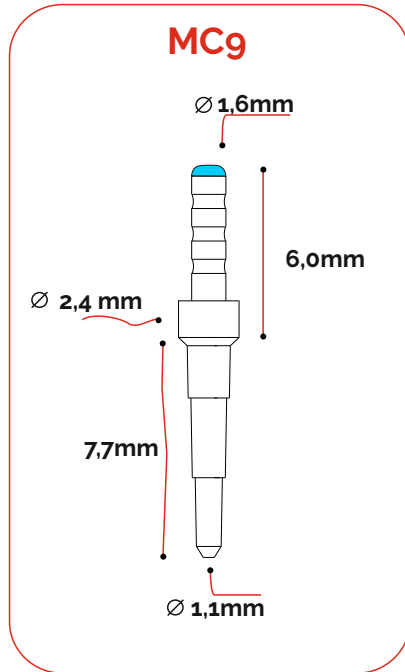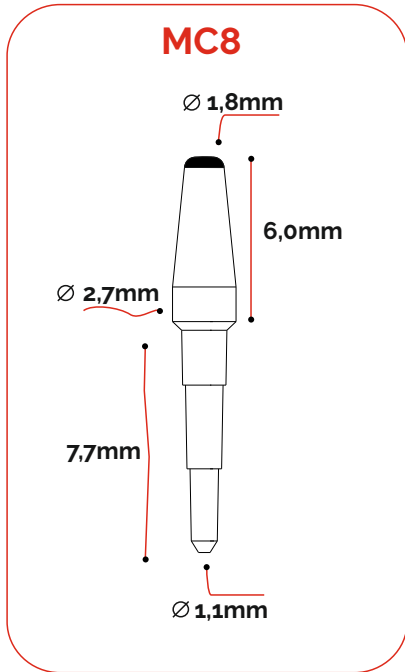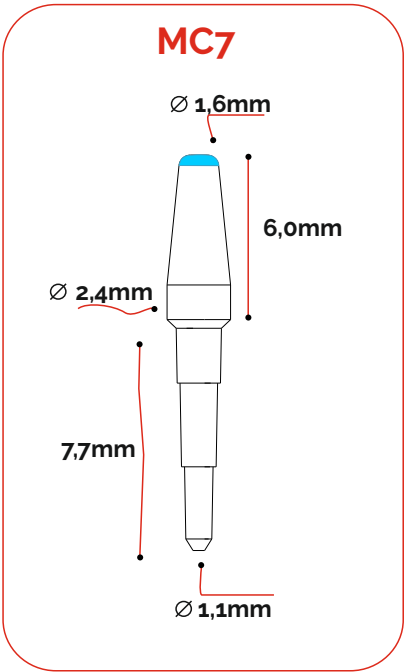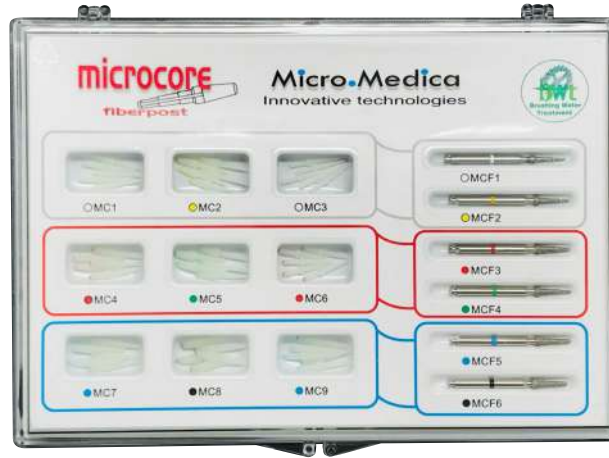
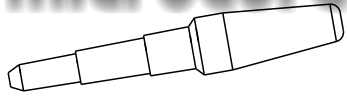
For those who wants to have all nine sizes of posts Micro Core is available a new kit that contains 3 posts for each size and the drills.
Each post head is colored with the same color as the drills to facilitate the matching.

Kit Introductivo (Perni e Frese) General Kit (Post and Drills)

CODE	BOX	PCS
MCFSET	01	45+6

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For each diameter is available a special drill, calibrated on the exact size of the post, the connection post drill is perfect. The drills Micro Medica are produced using high quality stainless steel that guarantees a high cutting power and excellent duration in time.

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AESTHETIC COMPOSITE CEMENT, DUAL HARDENING AND LIGHT-CURE

MICRO flowcore

DUAL CURE
FLOWABLE
MICRO FLOWCORE
CEMENT

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Micro Flow Core is an aesthetic composite cement, with dual hardening and photo hardening, for definitive cementation of dental restorations and for the reconstruction of prosthetic abutments. Thanks to its excellent fluidity and high micro hardness, it is particularly suitable for cementation of fiber endocanal pins.

Simple and efficient system. Optimal adhesion to dental tissue in combination with the new generation Micro Bond adhesive. Perfect connection with prosthetic reconstruction systems such as fiber and metal posts

Perfect aesthetics, Perfect for the cementation of inlays, onlays, veneers, and for all aesthetic works in general, thanks to a natural chromatic effect, and a stability of the color over time.

Easy removal of surpluses. In its initial pre-curing phase it is really easy to remove, thus guaranteeing surfaces perfectly clean

Resistant, Micro Flow Core shows a high resistance to micro fractures (320MPa) due to the high content of filler consisting of pyrogenic silica and glass.

Excellent Radio Opacity. The high radio opacity of Micro flowcore, makes it easier to detect any surpluses.

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The new line of micro strips and micro threads in glass fiber made in the Micro Medica laboratories, an innovative and easy to use product. Glass fiber today represents one of the most used reinforcement materials in dentistry. The micro strips and fiberglass threads are impregnated with light-cured composite and treated with a special silane liquid + adhesive to increase the adhesion of the composite to the glass fiber.

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With the new Micro Medica system, the fiber is ready for use, you just need to put it in position and light-cure it, high-performance reinforcements can be obtained perfectly in few seconds.

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The reinforcements of the new line impregnated with Flowable Composite have a significantly higher resistance than normal reinforcements treated only with adhesive, the composite penetrates deeply and impregnates evenly the glass fiber.

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Most of the similar systems on the market use as an impregnating agent for the glass fiber only the photo-curable adhesive so during the use the dentist or technician must add large amounts of composite to be able to finish the job.

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What you can do

Upper and lower teeth splinting, Orthodontic splinting,
Clinical applications, Space maintainers,
Stabilization of loose teeth, Provisional,
Maryland Bridge, Post trauma treatments,
Temporary bridges or single elements.

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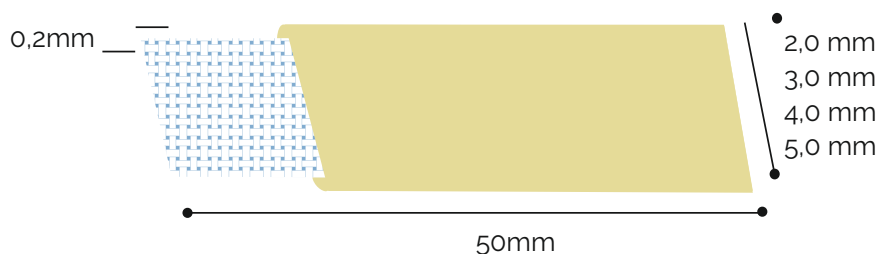
Bio Glass Strip and Wires, are a perfect tool for the reconstruction of missing dental elements, for the splinting of entire arches or for the realization of temporary crowns and bridges.

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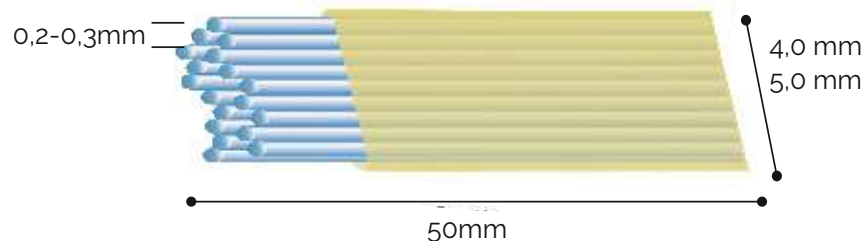
Micro Strips, multidirectional fiberglass tape prepreg with light-curing composite.

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Extracoronar splinting of dental elements that need to be stabilized, particularly indicated for lower teeth. The tape is extremely soft and adapts easily following the anatomical profile of the teeth on which it must be applied, to obtain high-performance locks it is recommended the use of several overlapping tapes.

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Micro Wires, unidirectional glass fiber wires prepreg with light-curing composite.

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Extracoronar splinting of dental elements with strong mobility, unidirectional wires offer excellent structural rigidity, they adapt easily following the anatomical profile of the teeth on which they must be applied.

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To protect our customers and their patients, Micro Medica's composites do not contain BPA derivatives that are believed to be harmful for the human organism.

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The composition of this composite system allows to realize all types of aesthetic restorations. The new matrix based on Urethane Dimethacrylate has been developed with the precise purpose of obtaining a highly performing material capable to faithfully replicate the colors of the Vita scale even in limited thicknesses.

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Zhefiro uniquely combines the advantages of cheapness and efficiency in a complete and compact system. A stable, aesthetic, high-tech and high performance material. Effective curing depth even with high thicknesses. Chromatic effect similar to the natural tooth. Stable and long-lasting colors. The wide spectrum of indications makes it effective for all types of aesthetic restorations, both for front and back teeth.

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Characterization of individual teeth to mask any discoloration.

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Our bleach composites were specifically developed for filling and restoring teeth bleached with modern bleaching systems.

The Dentin Bleach shades,

independent of the traditional shade system, are available in three shades, ranging from neutral to intense white, faithfully imitating the different shades of natural enamel undergoing bleaching treatments.

With these new shades, it is possible to precisely reproduce high-brightness restorations, ensuring high-quality esthetic results.

The Dentin Bleach shades, in paste and flowable form, can be combined with each other and are available in the following shades:

- Bleach 1,
- Bleach 2,
- Bleach 3.

In addition, the Intensive Supercolors can be used for internal and surface characterizations, to achieve optimal color integration with natural teeth.

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With Gingiva masses highly aesthetic gingival areas can be easily reproduced. Gingiva masses are independent from the chromatic system and are available in different color shades. The color shades range from light pink to purple, to be able to easily imitate the natural gum. Gingiva masses can be combined with each other, they are available in the colors: White, Light, Medium, Dark, Extradark. Furthermore, Intensive stains can also be used for internal and surface characterizations.

Indications;

- Imitation of natural gum
- Covering of long dental necks in bridge constructions
- Closing of large interdental spaces
- Modelling of areas with gingival involvement in implant structures
- Prosthetic superstructures of various kinds
- Characterizations of removable prostheses.

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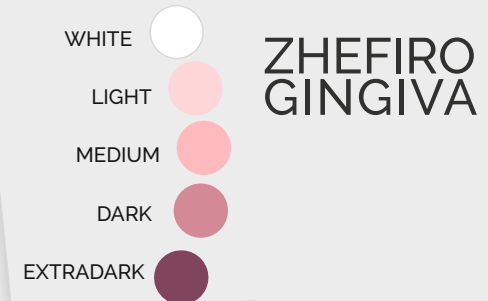
Characteristics;
High stability
Low plaque attachment
Natural aesthetic effect
Abrasion resistance
Excellent polishability

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5 GINGIVA MASSES

Trasparent Transparent Opalescent COMPOSITE

Versatile Aesthetic Different Unique

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Composite system for direct restorations.
Quick and easy reproduction of details, combinable with each other,
they are available in paste and flowable in the colors: transparent neutral and transparent opalescent

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The new nano filler gives aesthetic properties similar to the natural tooth.

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The high amount of inorganic filler guarantees an excellent mechanical resistance.

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Secure adhesion on all materials with specific adhesive system.

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Personalized restorations
with natural aesthetic effects

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Zhefiro Intensive is a photo-hardening flow composite suitable for the creation of direct highly aesthetic restorations. The Intensive fluid masses offer to the most demanding user the possibility of imitating in detail the colors and shades of the natural tooth.

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Zhefiro Intensive can be applied directly on natural teeth, on composite restorations, on resins or on digitally created works.

Zhefiro Intensive is perfectly suited to be used for internal or external modifications thanks to the new nano technology applied to it. With these fluid masses it is simple to simulate or mask fluorosis, it is possible to faithfully reproduce translucent incisal edges and fissure discolouration. Customs work of the highest quality.

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Super colors,

specifically designed for any aesthetic need.

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Modern composite with the new
spherical nano chroma filler

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Unicolor is all you need for an efficient, fast and dynamic workflow for the restoration of all classes cavities.

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Unicolor is the new revolutionary **monochromatic** composite capable to transform and adapt to the color of the natural tooth, a single product that covers all the colors of the Vita chromatic scale. Made using the new Micro.Medica nano chroma spherical filler, capable of absorbing the color of the natural tooth and replicating it the chroma perfectly. Proven quality and modern technology combined together give this product unique characteristics that stand out for their exceptional aesthetics, proven quality and high level of efficiency. The color-filled light that passes through the natural tooth is absorbed by Unicolor and as if by magic it transforms in color thus becoming an integral part of the tooth itself. With Unicolor it is not necessary to use complementary products, it can be used for fillings in both the anterior and posterior sectors, with a result unparalleled aesthetic.

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AESTHETICS

- aesthetic results in both the anterior and posterior areas
- natural-looking restorations in the anterior areas
- natural aesthetics in the posterior areas
- easy processing
- minimal susceptibility to fracture thanks to high flexural strength
- easy X-ray diagnosis thanks to high radiopacity

special with fiber quartz BioXhard.

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Bioxhard is an innovative material with characteristics very similar to natural dentin, used in combination with Zephro flow or pasta, guarantees perfect fillings, with almost double the mechanical resistance compared to traditional composites. The quartz fiber contained in Bioxhard prevents the propagation of microfractures that often form during the chewing phases. The fibers also guarantee, in addition to the chemical adhesion between the composites a high mechanical retention, ensuring perfect adhesion with any composite.

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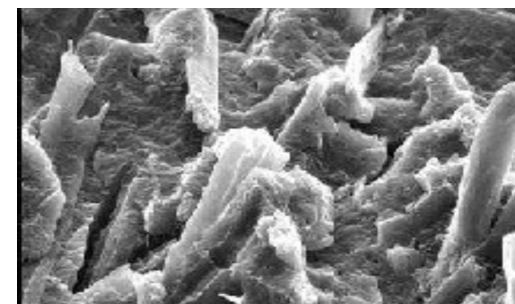
GUMFLEX

TEMPORARY FILLING

Nano-Hybrid Light Cure Composite Flowable

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GumFlex is a Nano Hybrid composite specific for temporary fillings of all types, this material allows an effective and safe temporary sealing, avoids infiltrations of any kind and preserves the cavities up to moment of final implementation. After filling, Gumflex remains elastic, and can be removed without problems even after a long period.

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Perfect in the Inlay-Onlay technique, it does not need to be removed with a bur, thus the cavity remains unchanged over time.

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Thanks to the ease of use of this new product, the obturation procedure is simple and fast, Gumflex however, it can be milled and polished if necessary.

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