metal free solutions

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Research and innovation are essential for Micro Medica to successfully enter the dental market and offer exclusive products to its customers. With thirty years of experience in the dental sector, our company has consolidated its position over the years and is now one of the leading manufacturers of fiber posts in the world. Not only posts, a significant portion of production is dedicated to special polymers and next-generation composites. This new experience with these materials has given a strong impetus to the development of the new department dedicated to carbon fiber, fiberglass, quartz fiber, and special hybrid materials. These materials are the basis of the new products the company has developed. New methods for producing prosthetics and structures using highly innovative computerized systems have given a strong impetus to research into alternative, biocompatible, aesthetic, and mechanically high-performance materials. Micro Medica's fiberglass and quartz fiber reinforcements have achieved great success. They are now the most widely used materials for dental reinforcements: their exceptional adaptability, ease of use, aesthetics, and above all, high mechanical strength have made them the perfect tool for achieving high-quality work. The practicality and ease of use of these materials makes it extremely easy to build high-quality prosthetics and opens up new possibilities for the dental laboratory. The new catalog is the perfect blend of cutting-edge technology and innovative materials capable of transporting dental technicians from the past to the future through a new way of creating increasingly cutting-edge and modern prosthetics. The company's commitment to developing new materials has opened the doors to technologies previously unknown in the dental world.

Micro Medica thanks everyone who made this possible.

Quality Certificate Uni Cei En Iso 13485

PERMANENT
NANOLITE
ULTRA

3D Printing Dental Composite
Long Term Implantable
Permanente / Definitivo

Crowns - Bridges
Prothesis on Implants

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NANOLITE 100% dental composite for 3D printing indicated for the production of permanent and definitive bridges and crowns, from single crowns to full arches, on natural teeth or implants.

Its physical and chemical characteristics guarantee the production of high-performance products with unparalleled aesthetics. Nanolite is a biocompatible material compliant with CE Standards, Certified Medical Device Class IIA.

The use of professional 3D printers is recommended. Available in all shades of the vita scale and in light, medium and dark pink.

Strength and Beauty in One Solution.

Thanks to our experience in composite manufacturing, we have developed Nanolite, a next-generation composite that perfectly balances strength and beauty.

The result? Prosthetics with a natural appearance and high performance, tailored to meet both clinical and aesthetic needs. A complete color range and outstanding visual quality make these materials ideal for any dental application.

Continuous Innovation and Evolution at the Service of the Customer.

Our materials are constantly updated to keep pace with market trends and meet the demands of dental technicians. Our goal is to simplify daily work by offering intuitive, reliable, and increasingly high-performance solutions.

Daicom

The new generation of composite material for build high resistance dental prosthesis

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DAICOM is the revolutionary system for the construction of prosthetic structures for implant rehabilitation. A perfect mix of composite and the most high quality sintered quartz fiber, designed to offer the best mechanical resistance of our artifacts, with an impeccable aesthetic with a drastic reduction of processing times.

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DAICOM FAST is created with fibers with a very high silica content that guarantee important mechanical properties. The structures made with this material have mechanical characteristics higher than conventional metal structures.

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DAICOM BRIDGE Unidirectional quartz fiber for reinforcing implant substructures, bridges, and crowns represents a true innovation in the dental field. The microfibers, bundled and impregnated with a special light-cured composite, are the starting point for creating incredibly rigid and high-performance substructures.

Daicom

The new generation of composite material
for build high resistance dental prosthesis

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DAICOM is the revolutionary material designed specifically for implant rehabilitation.

Bundles of sintered quartz fibers impregnated with composite ensure incredible mechanical strength and impeccable aesthetics.

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DAICOM is the revolutionary system for the construction of prosthetics structures for implant rehabilitation. A perfect mix of composite and sintered quartz fiber of the highest quality, designed to offer the best mechanical resistance of our products, with an impeccable aesthetic and with a drastic reduction in processing times.

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Daicom is created with fibers with a very high silica content that guarantee important mechanical properties. Structures made of this material have characteristics superior to conventional metal structures

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Resistance to compression

The obtained value of 1786 N with specific test, demonstrates how DAICOM is able to resist and withstand high chewing loads (Three Point Bending performed on Daicom bar, diameter of 5x4mm, length 35mm, distance between points support of 25mm.)

1 Estética natural de una estructura hecha de DAICOM
Natural aesthetics of a structure made of DAICOM

2 Prótesis fabricada con una subestructura metálica
Prosthesis made with metal substructure

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PERFECT polishing

It is a rebasable material
Daicom can be coated with composite or acrylic resin
Use with any technique or protocol
Perfect prostheses at a very low cost
Bisphenol Free
Biocompatible
Avoid phenomena of bimetallism, Metal Free
Immediate Loading
Photopolymerizable
Drastically reduces Processing times

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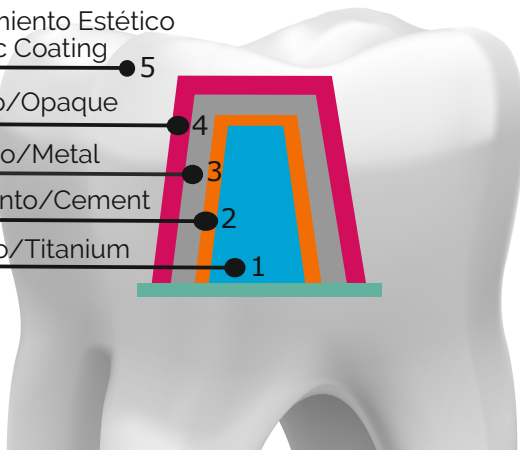
Discharge of forces on a surface reduced and thin: 5 different elastic modules, increase the possibility of breakage, detachments or infiltrations.


Discharge of forces over a large surface area and homogeneous: 2 different elastic modules, decrease the chances of breakage.

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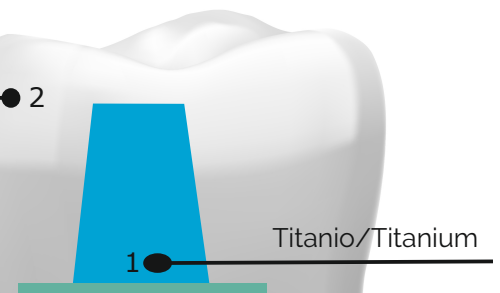DAICOM BRIDGE is the revolutionary system for the construction of prosthetics structures for implant rehabilitation. A perfect mix of composite and sintered quartz fiber of the highest quality, designed to offer the best mechanical resistance of our products, with an impeccable aesthetic and with a drastic reduction in processing times.

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Daicom Bridge is created with fibers with a very high Silica content which guarantee important mechanical properties. Structures made with this material have superior characteristics to conventional metal structures. A 100% made in Italy production, Unidirectional fibers for reinforcing substructures, bridges and crowns represent a real novelty in the dental field, the micro fibers collected in bundles and impregnated with a special light-curing composite, are the base to obtain incredibly rigid substructures e performers.

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Seguro / Safe

Económico / Cheap

PRODUCTS FOR DENTAL CAD CAM SYSTEMS

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In the field of dentistry, the adoption of a digital workflow is transforming the way we work and significantly improving the patient experience. Technological innovations are redefining diagnostic, therapeutic and communication processes, making work more precise, efficient and personalized.

Our materials are continuously refined to meet the changing demands of the market and dental professionals. The aim is to simplify daily work by offering intuitive, high-performing solutions that keep pace with modern dental workflows.

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CARBON FIBER AND GLASS FIBER

- CARBON FIBER DISCS
- FIBER GLASS DISCS

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PMMA 7 Layer Discs

Acrylite Multilayer 7 Layers Dentin/Enamel
The new standard for a realistic aesthetic, seven perfectly harmonized layers give a chromatic effect similar to the natural tooth.
The smart layer arrangement helps the natural passage of light, making our works unique and extraordinary.
Excellent match to the colors of the Vita scale.

Estabilidad cromática
Colour stability
Amplia gama de colores
Wide range of colors

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Smart Materials for Outstanding Results

Our next-generation materials combine mechanical strength and aesthetic properties in a single disc, enabling a wide range of applications. With a natural gradient of translucency—from dentin to incisal edge—they deliver restorations that are seamless, lifelike, and free of visible layering.

Strength and Beauty, Perfectly Balanced

Advanced manufacturing technology brings together the best of both worlds: durability and translucency. The result is restorations with a natural appearance and lasting performance. A full spectrum of shades and exceptional visual quality make these materials ideal for both functional and aesthetic needs.

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Acrylite is the new high-density PMMA resin developed in Micro Medica's laboratories. A high-performance material used for the production of discs for the CAD-CAM processing of bridges, crowns, mobile prostheses, toronto and even long-lasting temporary works. The unique features of these discs make them the perfect tool to make high-quality works and prostheses. Acrylite guarantees an accurate reproduction of every detail, even in complex restorations.

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Acrilite Multilayer Dentina Bleach

Superior aesthetics, guaranteed precision. Bleach discs stand out for their exceptional translucency and high natural fluorescence, achieved through an advanced bleaching process. Ideal for highly aesthetic restorations, they are the perfect choice for prosthetics with a natural and harmonious appearance.

Made from high-density multilayer PMMA, they ensure:

- Maximum accuracy in detail reproduction
- Excellent mechanical resistance, even at reduced thicknesses
- Easy processing and outstanding aesthetic results

Acrilite Multilayer Dentina Bleach is the perfect foundation for creating high-quality dental restorations, combining functionality, aesthetics, and reliability.

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Acrilite Monolayer Dentin

characterized by an exceptional translucency and a high natural fluorescence, they are the perfect tool to obtain high-performance prosthetic work. Made with the new monochromatic high-density PMMA that guarantees accuracy in the reproduction of details and resistance even in a small thickness. The ideal base for the realization of bridges, crowns and temporary works of great quality.

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**High-Density Transparent PMMA Discs
perfectly transparent, class IIA Biocompatible**

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High mechanical strength, low liquid absorption, corrosion resistance, long-lasting color stability, easy to mill, precise detailing, good structural rigidity. Ideal for making molds for molding light-curing materials. Indications: Face masks, surgical guides, bite splints.

PMMA Monolayer Pink Mobile Prosthesis Discs

Acrilite Monolayer Pink Mobile Prosthesis
 The ideal solution for the digital realization of prosthetic bases in a short time period and with a realistic monochrome effect. A new high-pressure production method guarantees this material an high mechanical performance, compact and with an high superficial microhardness it's the ideal tool to create perfect and long-lasting works.

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PMMA Bilayer Mobile Prosthesis Discs

Acrlite Bilayer Pink/Dentine Mobile Prosthesis, specific for the realization of mobile monolithic prostheses. The prostheses are milled with a single disc in only one process, reducing the need of manual processing. An impeccable aesthetic thanks to the new PMMA used for the realization of this disc. A perfect mix that allows you to create natural, personalized and amazingly detailed prostheses.

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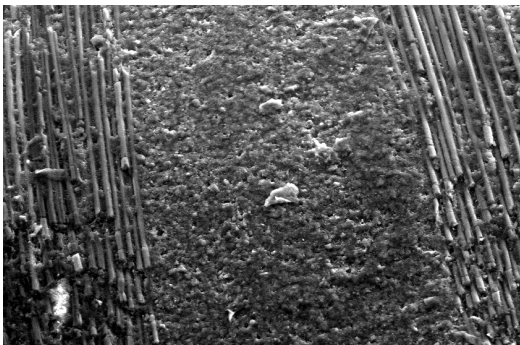
HI-TECH MATERIALS in Carbon Fiber and Glass
A new Hi-Tech technology developed in Micro Medica laboratories, allows to obtain fiber discs for dental applications with Cad Cam machines, Over 100 different layers of overlapped, pressed, and oriented fiber to obtain the maximum mechanical performance. A concentrate of technology, a combination of unique and innovative materials.

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

The photos taken with Sem show how the inside of the Bio Carbon discs is compact and free from defects. As shown by the study carried out by the Politecnico of Milan the carbon prosthesis milled from disc like the printed one, drastically reduces the stress transmitted during chewing to the implants by cushioning and dissipating homogeneously the forces generated.

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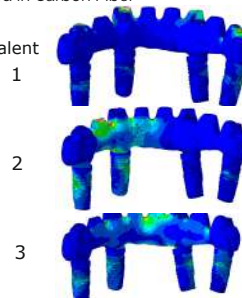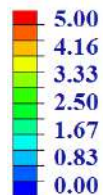Sfuerzo/Stress
Equivalent/Equivalent
[MPa]

CARBON and GLASS FIBER MULTILAYERS DISCS FOR DENTAL CAD CAM SYSTEMS

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Carbon fiber technical characteristics:

Breaking load 6000 mpa - Elastic modulus 180gpa
Density 1.4g/cm³ Specific weight 1.7gr/cm³
Water absorption <0.1%.

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The special fiberglass discs developed in Micro Medica allow the use of glass fiber for the realization of dental prostheses with Cad Cam Dental systems.

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Preformed arches in fiberglass and carbon fiber, available in two different sizes, bottom and upper and two colors, dentine and pink. With the use of a simple micro-motor and diamond or tungsten burs, temporary or definitive structures can be made in a very short time, with the same precision that can be obtained from a Toronto milled with CNC machines.

The used fibers have high performance characteristics from the mechanical point of view they have an hardness similar to acrylic resins, for this reason they are easily millable.

SUPER FIBER REINFORCEMENT

- BIO GLASS STRIP
- BIO GLASS DISCS
- MICROGRID
- 3D TCF TRIAXIAL
- CarboTex

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Glass fiber has now become an indispensable material in the daily routine of the dentistry field, its particular adaptability, ease of use, aesthetics and above all its high mechanic resistance, has made it a perfect tool to obtain high quality works. Indispensable for the reinforcement of structures for prostheses, bridges, crowns and inlays. This material has been used for many years successfully for multiple applications and is known and appreciated all over the world.

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Bisphenol Free

To protect our customers and their patients, Micro Medica composites do not contain BPA derivatives which are deemed harmful for the human body.

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Mechanical resistance of absolute relevance, fiberglass discs have an elastic modulus very similar to gold and noble alloys, they are a viable alternative to metal for the construction of structures in composite.

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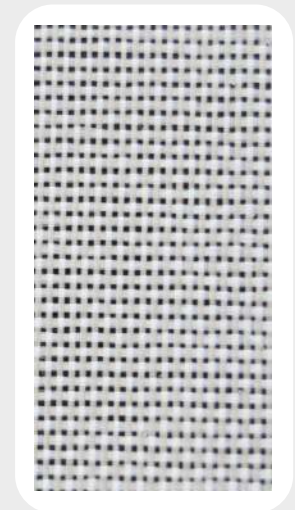
Super thin, with 0.05mm thickness, the Bio Glass Strip fabric is the thinnest on the market for this type of application, thanks to this particular feature it can be superimposed on several layers until a homogeneous and resistant structure is obtained, capable of withstanding important stresses.

The crossed fibers of this fiberglass fabric were designed to withstand vertical and horizontal stresses without suffering deformations or breaks.

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The fibers of these new systems are very soft and malleable, they adapt with simplicity to the model, and to the counter mold in seconds. They can build a lot of complex structures with the desired thickness by overlapping several layers of fiber.

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New in the dental field, born from the evolution and development of Micro Medica's glass fiber reinforcement systems, Bio Glass Disc is the missing piece of a technique that allows to realize substructures of bridges more or less wide, and of individual elements reinforced with fiberglass.

Detalle ampliado del tejido
Impregnado Bio Glass Special Discs
Enlarged detail of the fabric
Bio Glass Special Discs impregnated

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Virtually invisible, they are the perfect tool to reinforce structures of bridges and crowns both provisional than final.

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High Compatibility with all composites and acrylic resins in trade, the fibers are treated with special Silanes that increase adhesion, and are drowned in resin that become an integral part of it.

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Aesthetic, taking advantage of the features of this product you can realize works completely in composite or resin without having to resort to the use of opaque to mask the substructure.

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Glass fibers are capable of absorbing and dissipating loads of chewing. The structures made with this material despite being light and thin can withstand very high stresses.

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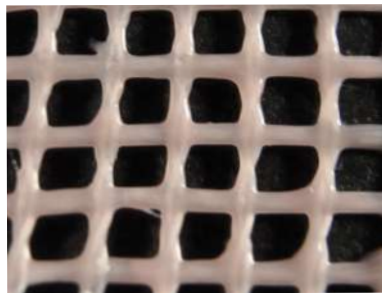
Micro Grid is the new Quartz fiber mesh impregnated with light-cured pink composite for the reinforcement and repair of mobile prosthesis. This product represents an important innovation that revolutionizes the traditional method in which metal meshes are used, which are absolutely incompatible with the acrylic resin of the prosthesis. With few and simple precautions, in few minutes the mesh can be incorporated into the prosthesis making it indestructible. The unique characteristics of this product makes it an economical, fast and overall a perfect tool to obtain a very high-quality reinforcements.

Biocompatible
CE Certified Medical Devices
in class IIa as long-term implantable.

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Micro Grid pre-impregnated
in pink composite

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Perfect Aesthetics

Thanks to its pink color, Micro Grid is practically invisible inside of the prosthesis

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Easy to use

Soft and malleable it fits with simplicity to the model and is finished in few seconds.

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
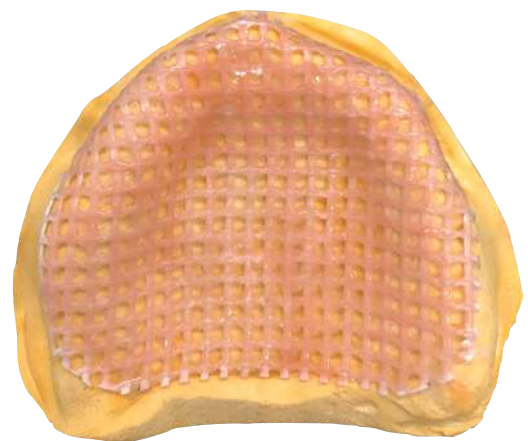
Thin, with less than a millimeter of thickness it fits perfectly to any type of prosthesis.

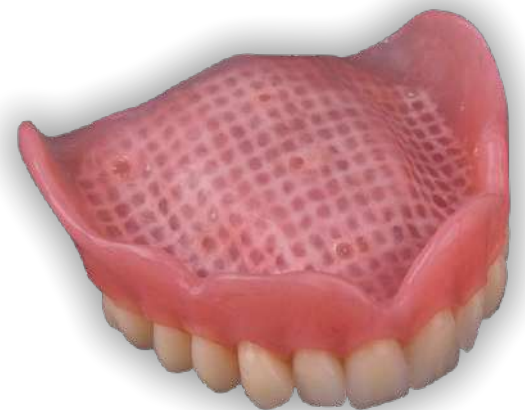Micro Grid fotopolimerizada
Micro Grid Light cured

Protesi Mobile con
Micro Grid inglobata

Mobile prosthesis with
incorporated Micro Grid

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Biocompatible, CE Certified Medical Devices in class IIa as implantable long-term.

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A new futuristic production system that allows the creation of an innovative 3D micro network using carbon fiber. This system is essential for the reinforcement of removable prostheses and it is also useful for the creation of light and resistant micro skeletons in acrylic resin.

After applying the opaque on the micro mesh it is easy to incorporate it into the prosthesis: an inseparable bond is created with the resin through the holes in the mesh, making our prostheses indestructible.

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Resistant, the prosthesis reinforced with Triaxial Carbon Micro Grid increases the breaking strength of mobile prostheses.

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Light, weighing less than a gram, it does not affect the final weight of the prosthesis and it does not create instability.

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Micro carbon mesh adapted and polymerized on the model

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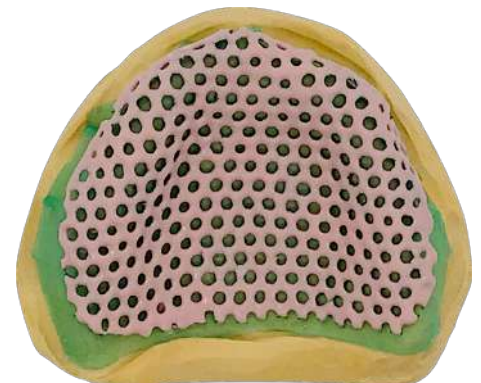
Hi-Tech, a technological reinforcement unique and innovative.

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Discharge of forces, the carbon fiber mesh is capable of absorbing and dissipating loads of chewing.

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
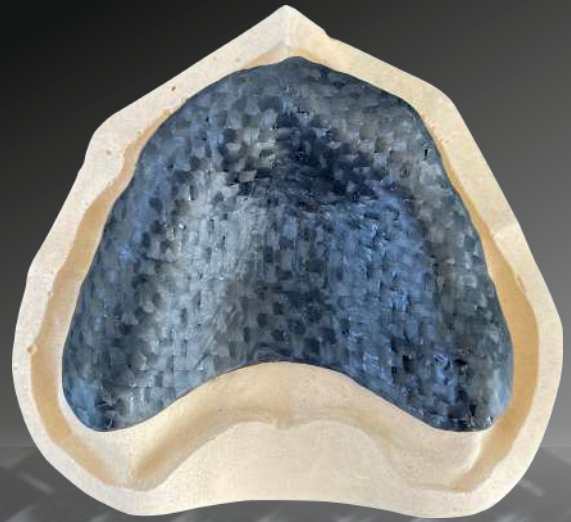
Biocompatible
CE Certified Medical Devices
in class IIA as long-term implantable.

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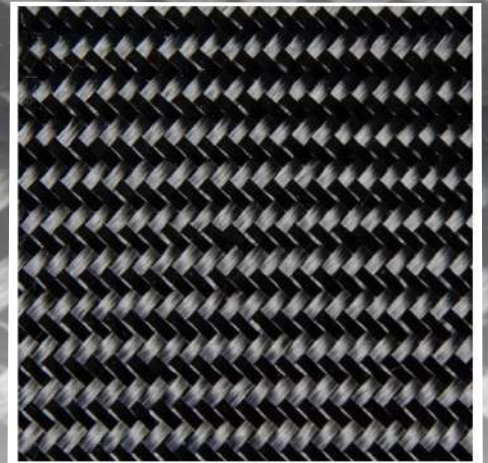The reinforcement with CarboTEX in the prosthesis increases exponentially its resistance to breakage.
Creation of micro skeletons in acrylic resin

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Discharge of Forces,
promotes the discharge of forces absorbing
and dissipating chewing loads.

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Lightweight,
weighing less than a gram

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Resistant,
the reinforced prosthesis increases significantly
its resistance to breakage

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Easy to use,
it adapts easily to the model

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Finally a great alternative to traditional methods.
A concentrate of technology in a few grams of weight.

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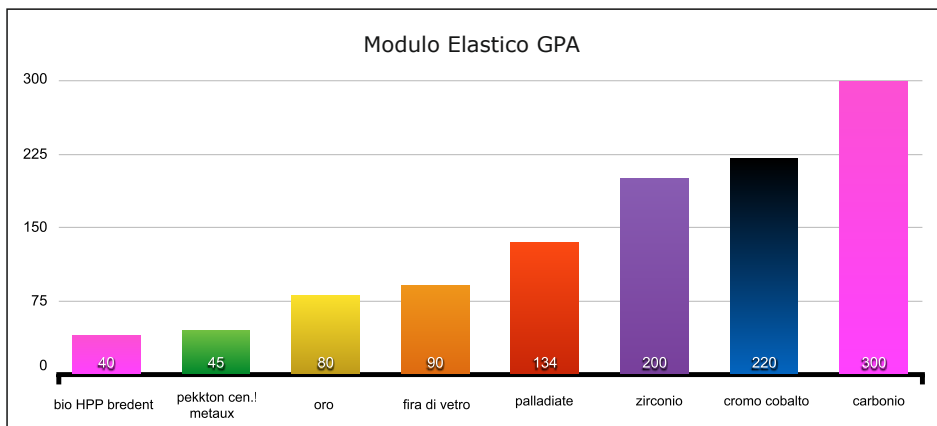
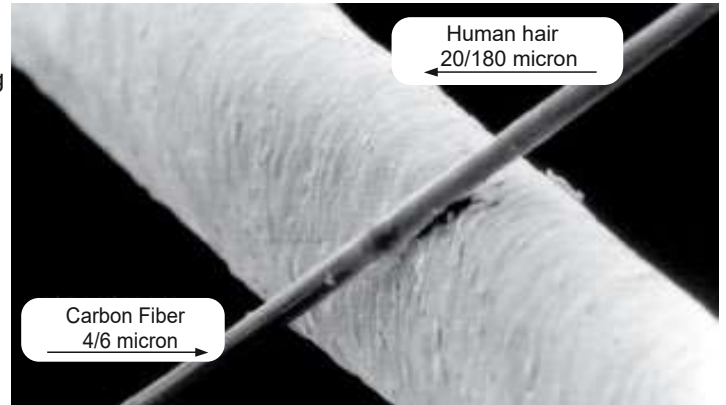
The new revolutionary Carbon Fiber system for the prosthetic reconstruction of Toronto on Implants developed in Micro Medica laboratories. Now it's possible use Carbon fiber also in the dental field and be able to appreciate the exceptional features of this product.

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This incredible material is used for multiple applications, especially in the aeronautical, mechanical and automotive; where lightness and extreme strength are required. By taking advantage of these important features Micro Medica has developed a special unidirectional carbon fiber made up of millions of fine threads (4/6 microns), this particular fiber was produced for the dental field. The new Bio Carbon Bridge system allows the realization of very extensive and immediately passive works, carried out directly on the plants without any contraction and with a surprising ease of use. The new carbon fiber designed by Micro Medica drastically reduces the stress transmitted to the implant in the chewing process, it also absorbs and dissipates the forces generated homogeneously.

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The Bio Carbon Bridge system in fiber of carbon, as shown in the graph reaches extremely high levels of mechanical resistance compared to other materials, which prove that this new product can replace metals and zirconia without any problems in the creation of dental structures. The thousands of filaments that make up the system allow the realization of even very complex substructures and extremely precise.

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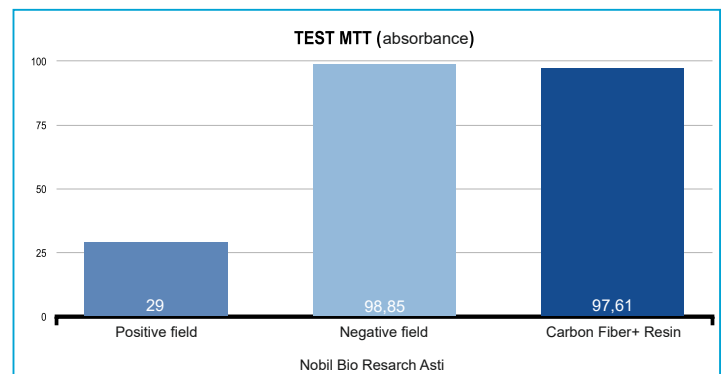
Light as a feather, with a specific weight of $1,4 \text{ gr} \times \text{cm}^3$. The Bio Carbon Bridge System in Carbon fiber is absolutely the material with weight-strength ratio between the best performing in the world. Compared to cobalt chrome 8.2 g cm^3 , 6.5 g cm^3 for zirconia, carbon fiber is much more light weight and efficient.

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Carbon fiber today is used with great success for many applications especially in the orthopedic sector, hundreds of prostheses built with this fantastic material are implanted every day. The tests of cytotoxicity carried out by the Nobil Research Laboratory Nobil Bio di Asti confirm that Bio Carbon Bridge is absolutely biocompatible. The Bio Carbon system Bridge is the first certified carbon fiber in the world for dental use.


The purpose of the research conducted by the Politecnico di Milano on the Bio Carbon Bridge Micro Medica is to establish, through one study of the finite elements, how the masticatory forces of different intensity are transmitted through the implant to the bone and through the prosthesis to the carbon fiber.

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Simulations FE

Boundary conditions and load mastication

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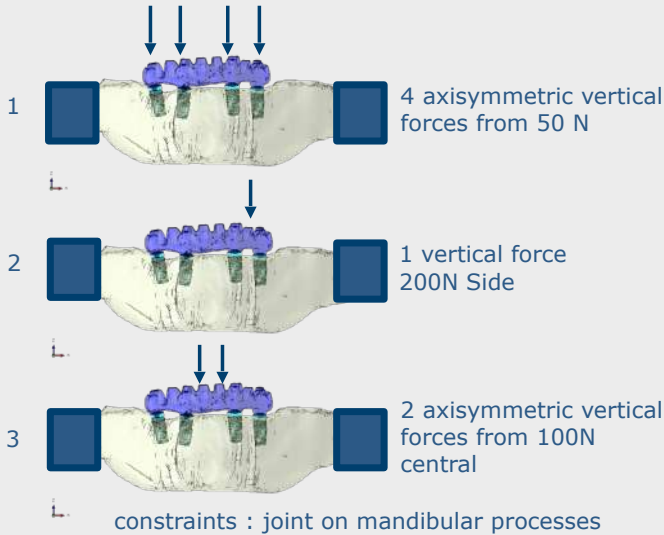Mechanical assessment of the effects of a Toronto Bridge structure carbon fiber through a studio in Finite Elements

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As shown by the study conducted by the Polytechnic of Milan, the new unidirectional carbon threads designed by Micro Medica, drastically reduce stress transmitted during chewing to implants, homogeneously dissipating the forces generated. This feature is extremely important for all work on implants.

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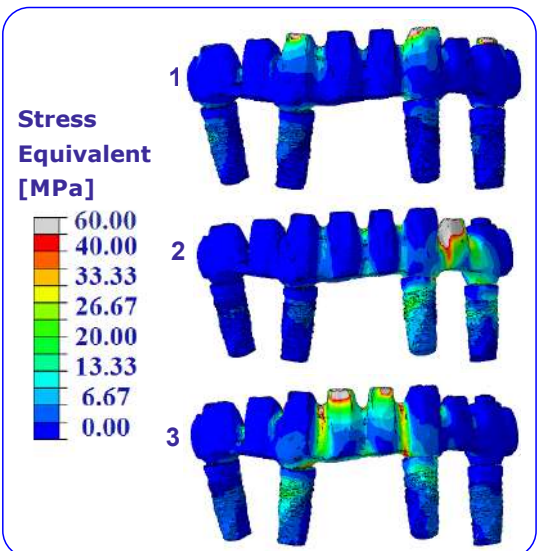
The structure of the "all on four" toronto model used for simulation it is the real scan of a case. A model was made after scanning virtual with prosthesis positioned at the level of the lower jaw. Considering three different axially symmetrical forces applied vertically in three different areas of the jaw, with loads ranging from 50 to 200N.

4 vertical forces of 50N for each level of the lower face
1 vertical force of 200 N at the level of the lower canine
2 vertical forces of 100N at each of the lower central level

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Carboblock is an adhesive system based on carbon fiber that allows the locking of the Carbon Toronto directly on the titanium abutments of the dental implant. The high adhesive power guarantees a perfect carbon-titanium connection, the locks are extremely performing and have a long life over time. Carboblock is the perfect solution to obtain high performance structures.

BiopolyCARB e BiopolyGLASS son Dispositivos Médicos de Clase IIA.
BiopolyCARB and BiopolyGLASS are a Class IIA Medical Devices

Tecno Polímero termoplástico de alta resistencia para subestructuras protésicas
High quality thermoplastic technopolymer resistance for prosthetic substructures

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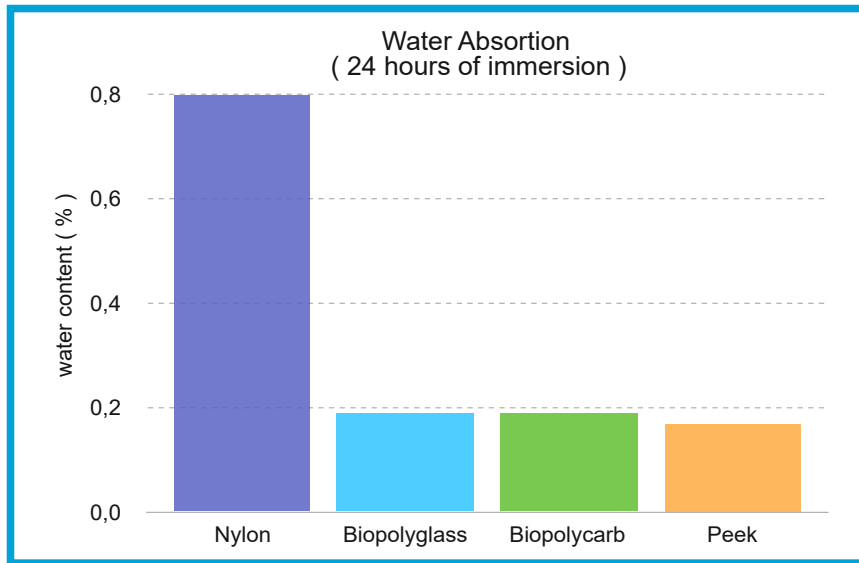
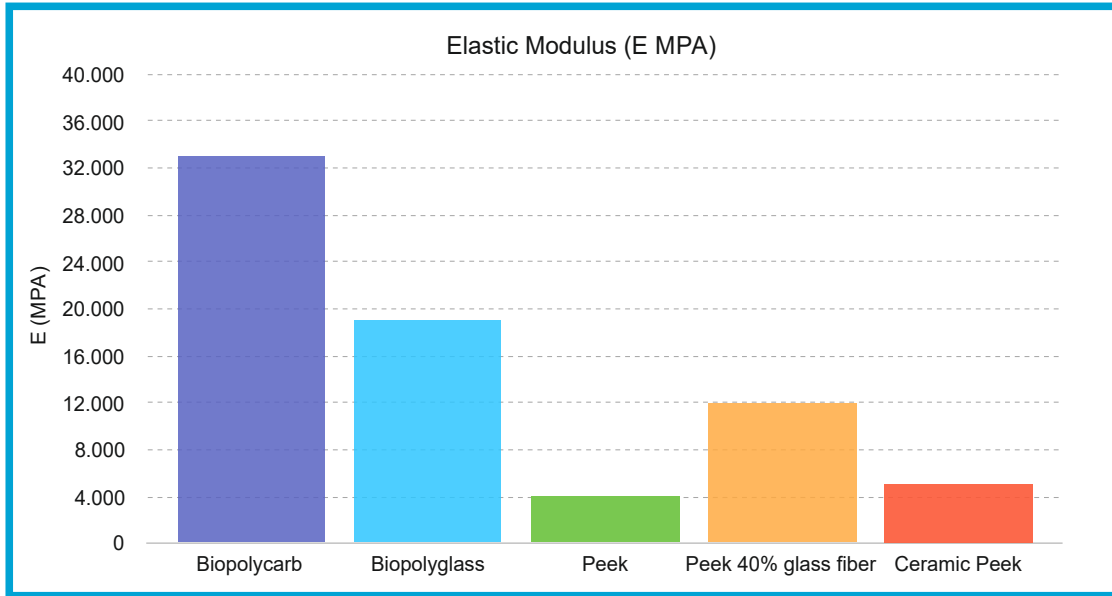
From the research of the Micro Medica laboratories BiopolyCARB and BiopolyGLASS is born, one new thermoplastic technical polymer loaded with special fibers, designed for the realization of high-performance prosthetic substructures. The combination of ultra-pure raw materials with microparticles of carbon fiber or glass fiber makes this product unique, a perfect blend capable of guaranteeing absolutely outstanding mechanical performance.

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Extremely high mechanical performance superior to any polymer currently on the market. Elastic modulus higher than 33000 Mpa for Biopolycarb. Particularly suitable for prosthesis on Toronto type implants. Compatible with all resins and composites for dental use.

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Water absorption less than 0.2%. Shrinkage from polymerization of less than 1%. Always passive jobs. Possibility to inject directly on the Master model. Usable with common nylon injection machines. Perfect aesthetics, Biocompatible.

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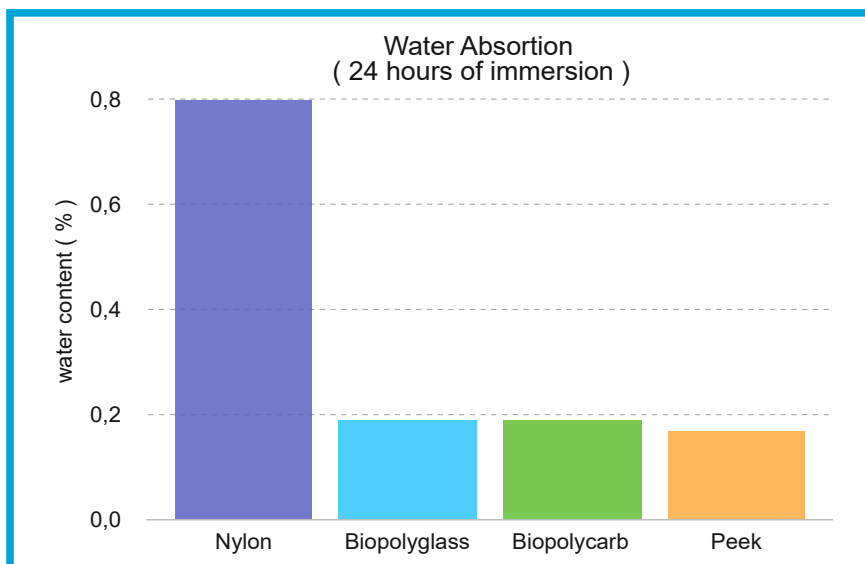
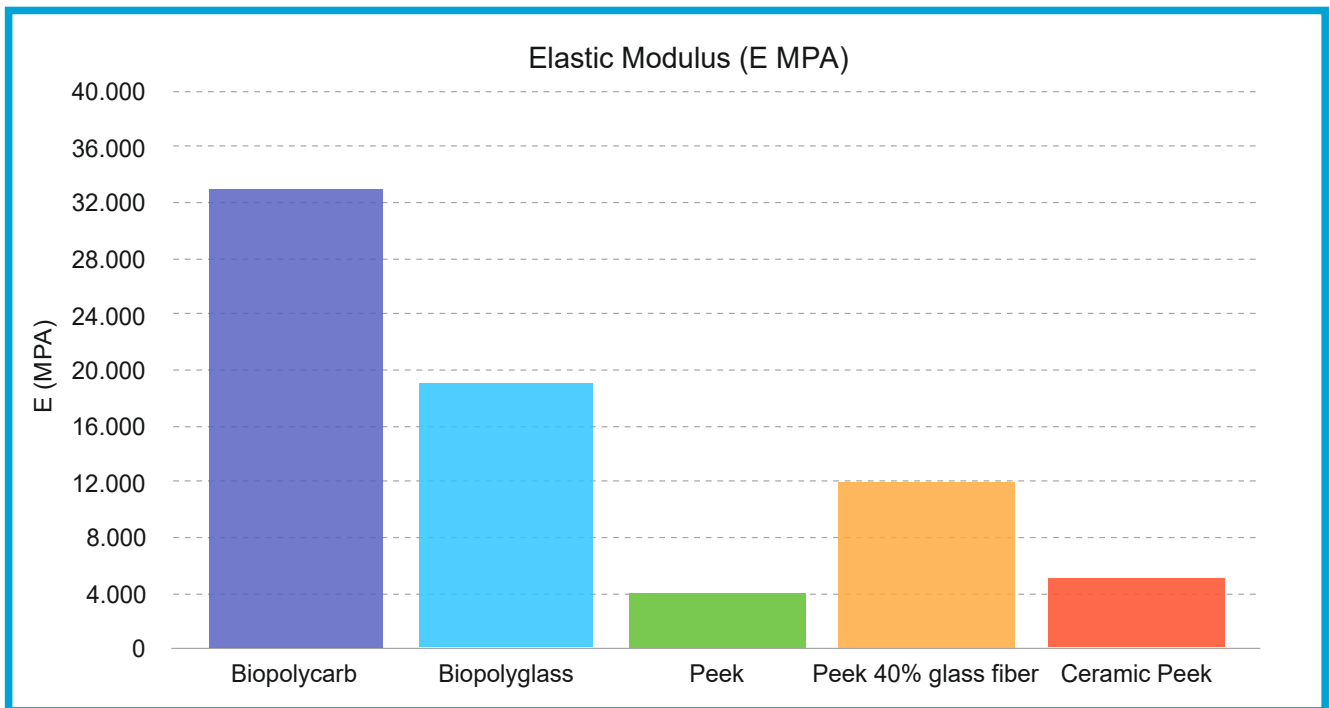
Extremely high mechanical performance superior to any polymer currently on the market. Elastic modulus higher than 19000 Mpa for Biopolyglass. Particularly suitable for prosthesis on Toronto type implants. Compatible with all resins and composites for dental use.

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Water absorption less than 0.2%. Shrinkage from polymerization of less than 1%.
 Always passive jobs. Possibility to inject directly on the Master model. Usable with common nylon injection machines. Perfect aesthetics, Biocompatible.

special with fiber quartz BioXhard.

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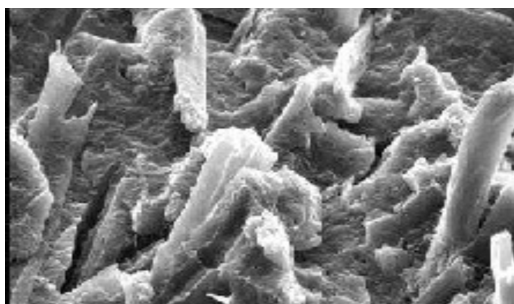
BioXhard is composite reinforced with quartz fiber

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BioXhard is an innovative material with characteristics very similar to natural dentin, used in combination with Zephiro flow or pasta, guarantees perfect fillings, with almost double the mechanical resistance compared to traditional composites. BioXhard can be used in the laboratory for the reinforcement of prosthetic structures. The quartz fiber contained in BioXhard prevents the propagation of microfractures that often form during the chewing phases. The fibers also guarantee, in addition to the chemical adhesion between the composites a high mechanical retention, ensuring perfect adhesion with any composite.

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MICROFAST Cement is a dual-use composite cement specifically designed for the needs of dental laboratories. A new formulation from UDMA distinguishes this new product, thanks to its excellent fluidity and micro-hardness. It is particularly suitable for all work requiring precision and high adhesion. **Ideal for cementing cobalt-chrome, titanium composites, and various types of resins.** Can be used in combination with the **new-generation Unibond adhesive** to obtain a simple and efficient system for perfect adhesion on all surfaces. During the initial pre-polymerization phase, it is very easy to remove, thus ensuring perfectly clean surfaces. **MICROFAST Cement** exhibits high resistance to microcracks(320MPa) thanks to its high filler content, using the new supranano technology that allows for homogeneous dispersion of the filler within the monomer.

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Natural color effect, and color stability over time.

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Micro Medica's new **Luminor Glaze** is made of light-curing UDMA composite with Supranano technology. It is the ideal tool to replace or complement light-curing methacrylate varnishes, in the daily routine of the dental laboratory. Its optical properties enhance the brilliance and shine of the artificial tooth, making it very similar to a natural tooth. The Supranano technology applied to this product gives prosthetics exceptional light refraction, making our work aesthetic and striking. Luminor does not yellow over time, is extremely durable, easy to use, does not create thickness, and is perfect for polishing and sealing characterizations created with flowable or liquid stains. Luminor can be used with composites and acrylic resins; it is perfectly compatible with PMMA and resins and composites for 3D printing.

a good job needs a perfect coating
high performance

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Light-cured supercolors for the characterization of indirect composite and acrylic restorations.

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Perfect Aesthetics
wide range of colors
vibrant shades
3D effect
transparency similar
to the natural tooth

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Long-lasting Results
perfect adhesion with acrylics
and composites high resistance
to fading steady over time
new generation polymer matrix

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Quick and Easy System
easy to use
ready to use colors
light-curing
abrasion resistant

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Blaze system
 code BLAZE8S
 pack containing 8 blaze bottles
 1 syringe of transparent flowable
 composite and accessories.

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Few steps to create high quality prosthetics.
 Replicating nature has never been easier.
 Bright colors and perfect surfaces will make our works
 stable and long lasting.

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Gel Coat is a one-component, photo-curable, completely hardening varnish, that forms a hard and compact surface without a dispersion layer. Chemically based on multi-functional composites and acrylates, great for sealing and polishing the surfaces of bridges, acrylic crowns or composite, full or partial dentures, carries individual impressions.

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Zhefiro Bonding one-component photo-curable adhesive agent, designed for surface treatment of prosthetic structures. The dispersion layer created after polymerization increases the adhesion between the layers of composite or acrylic resin on which it is applied. Particularly suitable for the treatment of carbon and glass fiber surface too.

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New Universal one-component adhesive
Innovative Chemical Formula,

UNIBOND makes it possible with a few simple steps to create the necessary conditions for perfect and long-lasting adhesion on many different materials.

UNIBOND is a specific 10Mdp-based adhesive for indirect restorations, it can be used for all types of metals, including precious ones, resins, composites, glass fiber, quartz fiber, carbon fiber and PMMA.

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Resin Primer is a specific adhesive for indirect restorations on all types of resins. This new single component adhesive allows, with a few simple steps, to create the necessary conditions for a perfect and long-lasting adhesion between acrylic resin-composite and composite on composite.

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Blue Gel is a vegetable gel used to inhibit the layer of superficial dispersion in dental composites. Applied to the surface of composite dentures before of the final photo-polymerization it drastically reduces the smear layer, which normally forms during the hardening phases by means of light. Blue Gel is compatible with all the composites on the market, it does not pollute and does not modify their physical and chemical characteristics. Easy to use, it can be removed from the surface of our works simply by washing them with water.

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Modelling Liquid composite is ideal to facilitate modelling and layering of photo-curable composites. The special formulation of this product makes it the perfect tool to obtain perfectly shaped surfaces. It does not pollute and does not alter the characteristics of the composites on which is used.

MicroBRIGHT

Sistema de pulido para composites,
nailon y resina

System for polishing
of composite, nylon and resin

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The base paste of the Micro Bright system is produced from pure raw materials and loaded with micro spherical particles. This unique and innovative formulation confers to this product the ability to finish and polish all dental resins with great simplicity.

Three differentiated granulometry pastes allow to obtain well-finished and shiny surfaces even on difficult to treat materials such as nylon. Using simple soft cotton brushes, you get really amazing results with great ease.

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You start using the Micro Bright paste n.1, this paste eliminates any type of imperfection on the surface of the prosthesis, making it perfectly smooth.

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The paste n.2 prepares the surface of the prosthesis to the final polishing, makes it shiny and uniform surfaces.

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The paste n.3, makes the surface bright and perfectly finished.

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For polishing the Nylon is necessary use a low speed 2000 rpm max whit a cotton brush very soft.

For polishing the Composite and the Acrylic resin is necessary use a speed between 1500-3000 rpm with cotton brush.

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WHAT YOU CAN DO

- Veneers
- Bridges
- Crowns
- Gingiva

Zhefiro Powder Opaque is suitable for covering dental alloys, carbon fiber, quartz fiber and resins. Due to its properties, the opaque contributes significantly to the creation of a stable bond and is suitable for the layering technique.

A specific advantage of the opaque is its very high stability during application, making the procedure easier and allowing for an accurate working technique. The opaque can be used flexibly for different application techniques with a brush or metal-free instrument.

A full range of complementary materials is available to assist you in the creation of custom colors and special color effects. The Zhefiro material system allows for natural color reproduction.

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Zhefiro Powder Opaque is available in the classic colors light, medium, dark and pink.

- Allows you to easily achieve the desired color
- Beautiful aesthetic results thanks to the natural optical effect
- Stability guaranteed
- Cost-effective restorations

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"To create highly aesthetic restorations and create stunning details and effects, we have created the perfect opaque»

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The indispensable gold color opaque for natural structures and with a chroma amazing

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Zhefiro Gold Opaque is suitable for covering base alloys, titanium and carbon fiber. Thanks its properties the opaque contributes significantly to the creation of a stable and lasting bond. Easy to apply, extremely covering and shiny. The opaque can be used with flexibly for various painting techniques application with brush or metal-free tool.

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A full range of complementary materials is available to help you create custom colors and special color effects.

CARBON FIBER

ZHEFIRO GOLD OPAQUE

- Allows you to easily achieve the desired color
- Perfect aesthetic results thanks to the natural optical effect
- Stability guaranteed
- Eliminates the gray effect on base metals

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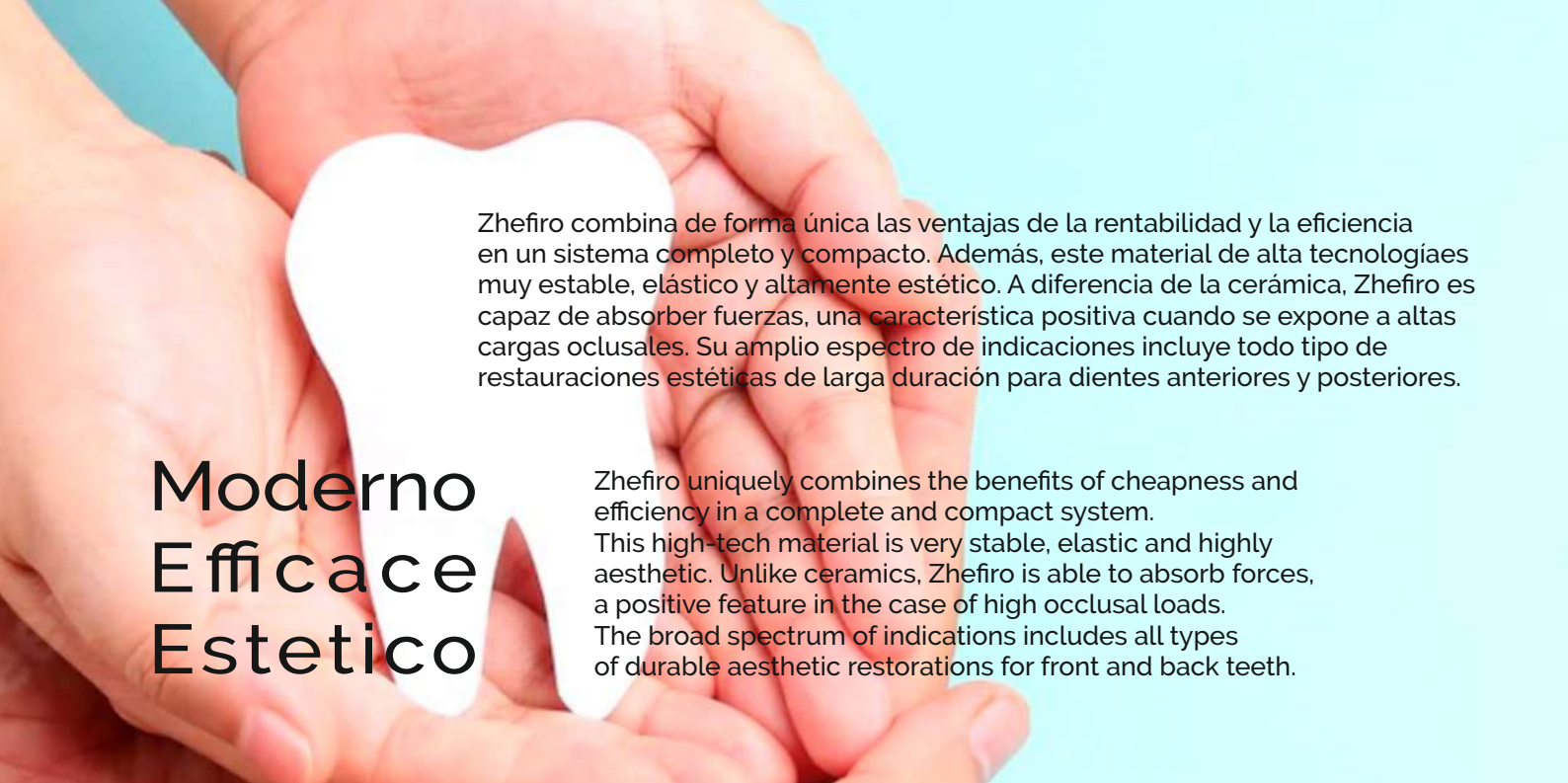
Composite system
for indirect restorations

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To protect our customers and their patients,
Micro Medica's composites do not contain BPA
derivatives that are believed to be harmful for the human organism.

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Moderno
Eficace
Estetico

Zhefiro uniquely combines the benefits of cheapness and efficiency in a complete and compact system. This high-tech material is very stable, elastic and highly aesthetic. Unlike ceramics, Zhefiro is able to absorb forces, a positive feature in the case of high occlusal loads. The broad spectrum of indications includes all types of durable aesthetic restorations for front and back teeth.

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The composition of this composite system allows to realize all types of aesthetic restorations in the front and back sectors of any structure. Even in limited thicknesses Zhefiro is able to faithfully replicate the colors of the Vita scale .

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A complete system for all indications

Coronas y puentes totalmente recubiertos
Fully coated crowns and bridges
Estructuras y superestructuras telescópicas
sobre implantes totalmente recubiertos
Telescopic structures and superstructures on
totally coated systems
Trabajos con aditamentos / Work on attacks
Recubrimientos oclusales/ Occlusal coatings
Inlay / onlay
Carillas / Veneers
Provisionales a largo plazo / Long-term temporary
Caracterización de dientes protésicos
Characterization of teeth for prostheses

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Zhefiro Labor composite is the result of advanced research in the field of high-performance aesthetic materials. Thanks to the Layering Technique, this composite allows for the creation of highly aesthetic, precise, and functional products, ideal for applications where visual appeal is as important as strength and reliability. The versatility of Zhefiro Labor allows complete control over the color transitions, depth, and naturalness of the final product, adapting perfectly to different clinical and technical needs. Below are some practical examples of layering and use of the various Zhefiro Labor composites, designed to maximize each specific case. These diagrams demonstrate how, through careful and personalized application, it is possible to achieve excellent results in terms of aesthetics, durability, and integration.

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Effects available in the Zhefiro Labor range:

- Bleach / White – incisal brightness
- Blue / Gray – for translucent effects
- Amber / Ocher – internal color intensification
- Brown – deep areas and furrows
- Opalescent / Transparent – dynamic light reflections

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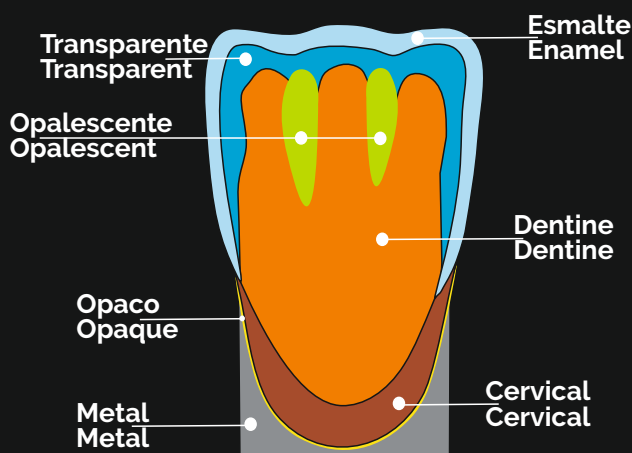
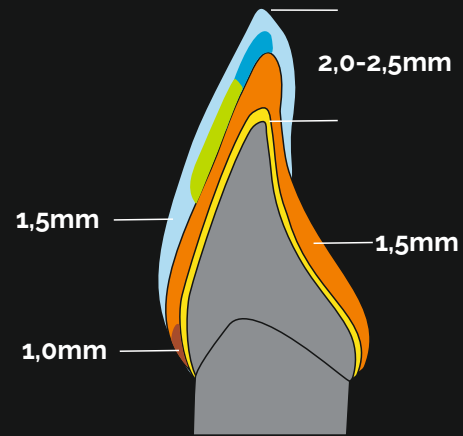
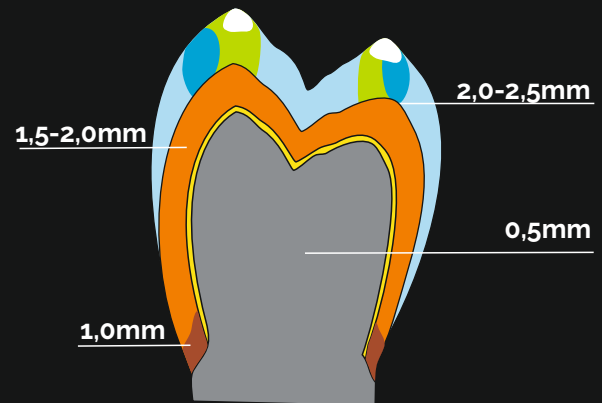
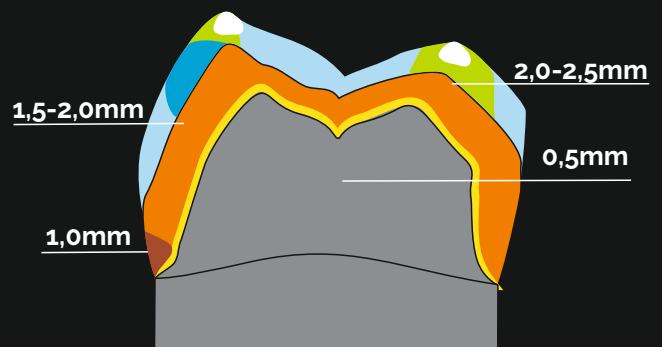
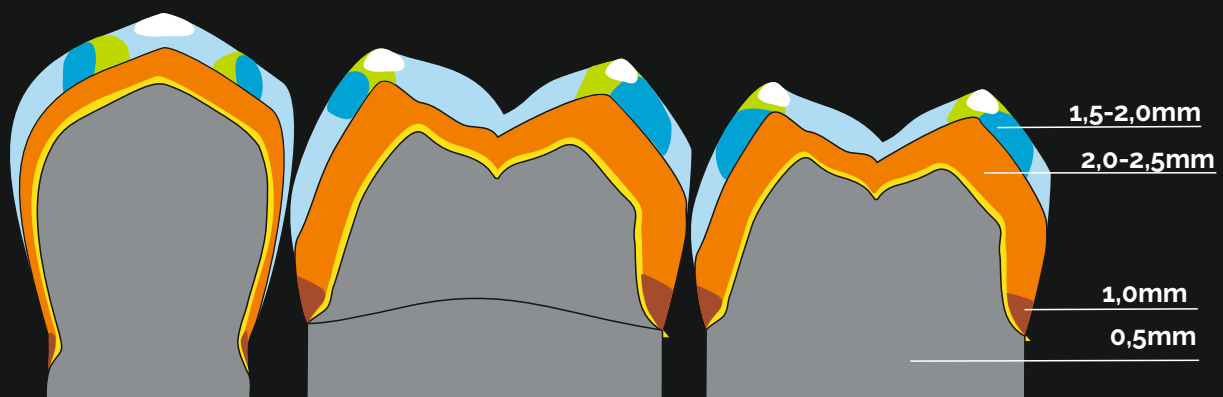
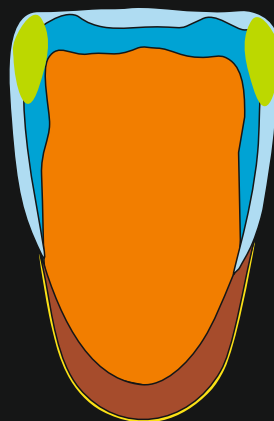
Thanks to its versatile color palette and workable consistency, Zhefiro Labor is perfectly suited to any type of restoration, offering technicians and clinicians a complete tool for achieving precision, aesthetics, and functionality.

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Zhefiro flowable Opaque is a photo-curable new generation Opaque, based on UDMA resin, thanks to its characteristics It is able to perfectly mask under dental structures made of metal, carbon fiber, quartz fiber, glass fiber and resin.

Its very high covering power makes it indispensable for creation of highly aesthetic works. Ready to use, it is available in dentine and pink colours.

The material properties allow for quick application and help to determine the final color of the covering.

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Zhefiro Intensive is a photo-hardening flow composite suitable for the creation of indirect highly aesthetic restorations. The Intensive fluid masses offer to the most demanding user the possibility of imitating in detail the colors and shades of the natural tooth.

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Zefiro Intensive can be used for the creation of composite structures, resin or digitally works.

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Super colors, specifically designed for any aesthetic need.

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Zefiro Intensive is perfectly suited to be used for internal or external modifications thanks to the new nano technology applied to it. Customizing individual elements or extended works has never been so simple, a wide choice of colors to satisfy any aesthetic need. Customs work of the highest quality.

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With Gingiva masses highly aesthetic gingival areas can be easily reproduced.

Gingiva masses are independent from the chromatic system and are available in different color shades.

The color shades range from light pink to purple, to be able to easily imitate the natural gum.

Gingiva masses can be combined with each other, they are available in the colors: White, Light, Medium, Dark, Extradark, Flamingo, Salmon, Plum, Faded Cherry, Cherry and Deep Cherry.

Furthermore, Intensive stains can also be used for internal and surface characterizations.

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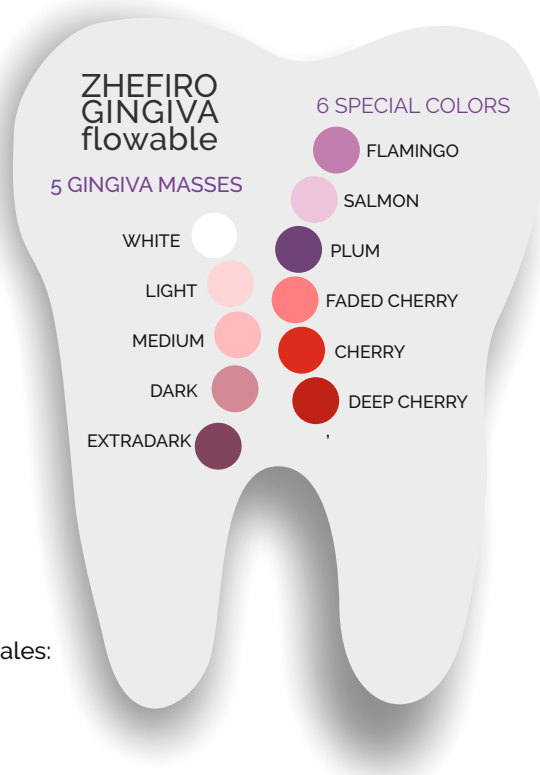
- FLAMINGO
- SALMON
- PLUM
- CHERRY
- FADED CHERRY
- DEEP CHERRY

Characteristics:

- High stability
- Low plaque attachment
- Natural aesthetic effect
- Abrasion resistance
- Excellent polishability

Indications

- Imitation of natural gum
- Covering of long dental necks in bridge constructions
- Closing of large interdental spaces
- Modelling of areas with gingival involvement in implant structures
- Prosthetic superstructures of various kinds
- Characterizations of removable prostheses.

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With Gingiva masses highly aesthetic gingival areas can be easily reproduced. Gingiva masses are independent from the chromatic system and are available in different color shades. The color shades range from light pink to purple, to be able to easily imitate the natural gum. Gingiva masses can be combined with each other, they are available in the colors: White, Light, Medium, Dark, Extradark, Faded Cherry, Cherry and Deep Cherry. Furthermore, Intensive stains can also be used for internal and surface characterizations.

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Characteristics:
High stability
Low plaque attachment
Natural aesthetic effect
Abrasion resistance
Excellent polishability

Indications

- Imitation of natural gum
- Covering of long dental necks in bridge constructions
- Closing of large interdental spaces
- Modelling of areas with gingival involvement in implant structures
- Prosthetic superstructures of various kinds
- Characterizations of removable prostheses.

3 colores especiales:
FADED CHERRY
CHERRY
DEEP CHERRY

 **LABOR**
ZHEFIRO

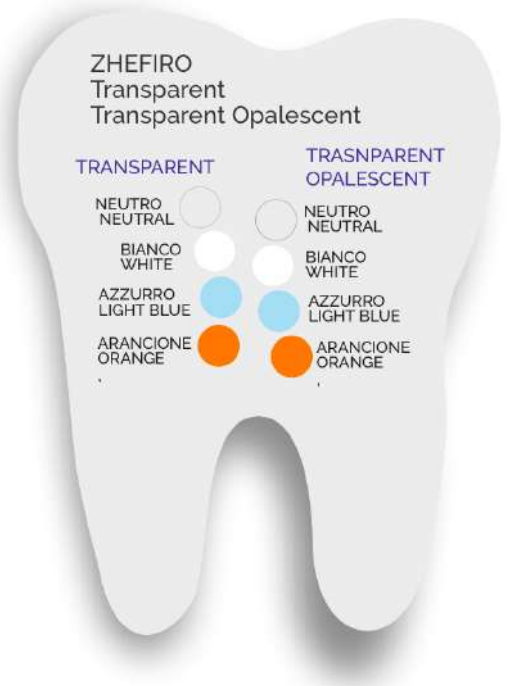
**Trasparent and Opalescent
COMPOSITE**

Versatile
Aesthetic
Different
Unique

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Composite system for indirect restorations Quick and easy reproduction of details, combinable with each other, they are available in paste and flowable, in the colors: neutral, white, light blue and orange.

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What you can do

Front teeth fillings, Back teeth fillings, Characterization of single teeth to mask any discolorations, Veneers, Inlays / onlays, Incisal reconstructions, Occlusal coatings, Bridges and crowns, Fully veneered telescoping and implant supported, superstructures, Characterization of prosthetic teeth, Fully veneered crowns and bridges, Attachment work, Occlusal veneers, Long-term temporaries.

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Secure adhesion on all materials with specific adhesive system.

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The high amount of inorganic filler guarantees an excellent mechanical resistance.

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The new nano filler gives aesthetic properties similar to the natural tooth.

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Our bleach composites were specifically developed for the creation of aesthetic veneers, bridges, and crowns that can be integrated into patients who have undergone teeth whitening using modern bleaching systems.

Bleach Dentin shades, independent of the traditional color system, are available in three shades, ranging from neutral to intense white, allowing them to faithfully imitate the different shades of natural enamel undergoing bleaching treatments.

With these new shades, it is possible to precisely reproduce high-brightness restorations, guaranteeing high-quality aesthetic results.

Bleach Dentin shades, in paste and flowable form, can be combined with each other and are available in the shades: Bleach 1, Bleach 2, Bleach 3.

Intensive super shades can be used for internal and surface characterizations, to achieve optimal color integration with natural teeth.

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TRANSPARENT FLASK FOR PHOTOPOLYMERIZABLE COMPOSITES.
Completely transparent or with a decomposable aluminum base, MASTER flasks allow you to work faster and more economically thanks to the presence of reducers. MASTER flasks are composed of transparent elements to optimize exposure to ultraviolet rays.

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Special Flask in Aluminium and Plexiglas

The Master flask arises from the need to have an instrument perfect for all new molding techniques. To create a precise and reliable tool, all the pieces were made by milling from solid. The transparent lid allows light to pass through the flask without encountering obstacles, in this way perfectly hardened composites can be obtained. The steel guide bushings guarantee the perfect sliding of the cover on the base of the flask, and the ergonomic wing nuts make it easy to close. Perfect coupling with the internal reducer.

Reductor grande para la mufla MASTER
Big Reducer for Flask MASTER

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With the Master flask it is possible to create prostheses of any size, the transparent cover is equipped with an internal reducer that allows a large saving of transparent silicone.

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Detail of the threaded bushings and sliding guides

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Reductor pequeño para mufla MASTER1
Small Reducer for Flask MASTER1

Muffola/Flask MASTER1

The Master1 flask is particularly suitable for the manufacturing of small crowns and bridges. Made with great care and with high quality materials, it is the indispensable tool to obtain printed prostheses of the highest level.

The special transparent reducer is also available for this muffle, which allows you to save an important amount of silicone.

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New Fluid Silicone from laboratory 100% transparent with high final hardness over 72 Shore.

Developed for light-curing aesthetic materials, accurate waxing detection and perfect reproduction of details.

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Another important step in the search for innovative materials, Micro.Medica has made the new high temperature resistant silicone with nano filler Ceramic. This material has been designed to be used with everyone the new thermoplastic polymers. With Thermosil it is possible to make the mold of the silicone flask, filling the same counter mold with plaster. In this way, once the polymer has been injected directly into the silicone, without problems you will get precise pieces free from any kind of impurities.

MICRO.MEDICA S.R.L.
VIA ROBBIO 18B, 27030 PALESTRO (PV) ITALY
e.mail info@micromedicasrl.it
website www.micromedicasrl.com

Rev.10.2025